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AQUA- Pressure Pot

Pressure polymerization pot for cold curing polymer

1. GENERAL

1.1 This operation manual allows a safe and efficient handling of this unit.

The operation manual is part of the unit and has to be in close proximity of the unit and always accessible for the staff members. The staff members have to read and understand this manual before starting any work with this unit. The basis for safe working is the strict observance of all mentioned safety information and directions in this manual. Beyond that the local accident prevention regulations and general safety regulations for the application field of the unit are in force.

1.2 Limits of liability.

The manufacturer does not assume any liability for damages due to ...

- Disregard of the operation manual
- Inappropriate use
- Employment of unskilled staff members
- Unauthorized conversion of the unit
- Technical modifications
- Use of non-licensed spare parts

Technical alterations due to improvement and further development are subject to modifications.

1.3 Copyright protection.

This operation manual is copyright reserved.

Transferring this manual for use to third parties, duplication in any way and manner – even in extracts – as well as use and/or transfer of the content are prohibited without the manufacturer's permission in written except for internal use.

1.4 Spare parts.

Attention: False or faulty spare parts can affect safety and lead to damages, hazardous malfunctions or complete failure.

- Only use original spare parts of the manufacturer.

1.5 Warranty regulations.

We guarantee this product against all material and manufacturing defects for one year – thereof excluded are: wearing parts, gaskets and defects caused by inappropriate handling.

2. SAFETY

2.1 The operator's responsibility.

The unit is exclusively used in the commercial field. Due to this fact the operator of the unit is subject to the legal obligations for safety at work. The operator has to make sure that all staff members using this unit have read and understood the operation manual. The unit needs to be used by trained and qualified personnel only.

2.2 Appropriate use.

The unit is exclusively designed and constructed for the use herein described: The AQUA-Pressure Pot is made of stainless steel and is suited for a homogeneous and bubble-free polymerization of autopolymerizates. When using our pot in connection to our AQUA-heating plate (order no. 581-009-007), the desired temperature in the pot can be retained.

2.3 Extraneous Hazards

The herein mentioned safety notes as well as the warning notes mentioned in further chapters of this manual have to be observed in order to reduce health hazards and to avoid dangerous situations.

Never use the AQUA-Pressure-Pot for cooking dentures or any other objects not indicated. Never heat it over 60°C (140 degree Fahrenheit)!

For damages due to too high performance any liability will be disclaimed!

NEVER use gas or similar non-adjustable (non controllable) heaters!!

- Never leave the pressure-pot unattended during use.
- Pay absolute attention to the fact that the lid is correctly situated on the pot and correctly closed, just as described in this operating instruction.
- Only if both handles (handle of pot (lower) and lid (upper)) are positioned exactly one above the other, the pot will be correctly closed and can be pressurised.
- Prior to opening the pot, attention must be paid to the outlet valve (7) being opened: lever (8) in a vertical position and the pressure being completely released (manometer = „0“).
- The operating pressure should not exceed 2 bar. The safety valve (4) automatically releases pressure when exceeding 2.2 bar.
- The safety valve (4) should be checked regularly (at least once per month). Pressurise the pot. The safety valve has to ventilate at approx. 2.2 bar. If this is not the case, the safety valve has to be exchanged.
- Due to safety reasons we advise an inspection of your pressure pot (depending on operational demands) in our premises every 2 years.

2.4 Care and maintenance

- We recommend a weekly cleaning of the interior pot to eliminate lime and acrylic particulates.
- Do not use acrid and scouring agents as well as sharp-edged and pointed instruments, since they may damage the surface.
- Please pay attention to the fact that sharp-edged flasks, articulators, etc. can destroy the surface.
- Operating the pot under temperature without water leads to damage of the mechanical parts as well as of the seal.
- Please keep the lid's seal always clean and free of grease. If necessary use talcum.

3. DELIVERY CONTENTS

| | | |
|------------------------------|-----------|-------------|
| Filling hose including valve | order no. | 591-009-006 |
| AQUA heating plate | order no. | 581-009-007 |

4. INCOMING CONTROL

Please check the unit and accessories immediately after receipt for probable transport damages and defects. Warranty claims can only be enforced if the vendor or shipper is informed immediately. Then a document of damage has promptly to be made up. The document of damage has to be sent directly to your **al dente** deposit, for that the claims of compensation can be sent to the insurance. When returning the unit to an after-sales **service of al dente**, please use the original packaging. The following papers are to be added: Name and address of the owner, number of the device (see type plate), description of the defect.

5. OPERATING THE AQUA- Pressure Pot

5.1 Bedienungselemente

- 1 Upper handle
- 2 Lower handle
- 3 Plate slide
- 4 Safety valve
- 5 Filling valve
- 6 Manometer
- 7 Outlet valve
- 8 Lever

| | 4,5 liters | 7,0 liters |
|-----------------------|--------------|--------------|
| Inner diameter | 220 mm | 220 mm |
| Inner height | 110 mm | 210 mm |
| Operating pressure | max. 2 bar | max. 2 bar |
| Working overpressure | max. 2,2 bar | max. 2,2 bar |
| Operating temperature | max. 60°C | max. 60°C |
| Weight | 2,2 kg | 3,2 kg |

5.2 Relevant information for operating the unit.

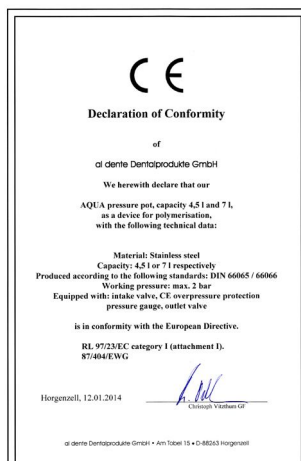
5.21 Pressurising the AQUA-Pressure Pot

- Put the objects to be polymerized into the pot and fill in water until your object is completely covered.
- Position the lid in such a way to the pot's rim that the marking on the lid (O) and on the lower handle (2) (O arrester fitting) are opposite to each other.
- Press the lid evenly down on that side that is opposite to the upper handle (2). While pressing down swivel the upper handle of the lid (2) at the same time to the left clockwise until it is exactly positioned over the lower handle of the pot (1).
- Close the outlet valve (7) well: lever (8) in a horizontal position
- Attach the pressure hose to the filling valve (5) and pressurise the pressure pot with 2.0 bar operating pressure. Reaching a pressure of more than 2.2 bar, the safety valve (4) releases pressure automatically.
- The manometer (6) indicates the operating pressure.
NEVER use gas or similar non-adjustable (non controllable) heaters!!
Never heat the AQUA Pressur Pot over 60°C (140 degree Fahrenheit)!

5.22 How to open the AQUA-Pressure Pot

- Open the outlet valve (7): lever (8) is in a vertical position – pressure is released.
- The manometer (6) must indicate a pressure of 0 bar.
- Pull the plate slide (3) back as far as it will go and at the same time turn the upper handle (2) anticlockwise until the marking of the lid and of the lower handle (1) are positioned one upon the other.
- Take off the lid from the pressure pot.

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al dente Dentalprodukte GmbH
• 27012014 • © by al dente dentalprodukte GmbH