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Indications for use

Dental thermoforming blank for the production of occlusal splints.

Material description

Super-transparent, stringily elastic thermoforming blank made from polyethylene terephthalate glycol for the production of dental splints. Bonds with acrylate.

Field of application

Suitable for the production of bases for occlusion splints and units used in orthodontics, for hard occlusion splints, medication splints, orthodontic devices as well as band-age plates.

Processing

All Biolon materials are delivered with an insulating foil. This extremely thin foil (0.07 mm) is thermoformed together with the Biolon on the model side and guar-antees a decisively improved transparency of the splint. Thus an additional insulation of the model surface is no longer necessary.

If processed in a Vacfomat, the insulation foil must be removed before thermoforming.

The adjustment of the heating and cooling times can either be taken from the survey of thermoforming foils, or can be programmed directly into the unit with the help of the barcode on the Biolon packaging.

If a Druformat scan or a Drufosmart scan is used, please take note of the special information included in this package. We recommend Cenit L ([REF](#) D3815) for the necessary blocking out works that are done on the model.

Protect against humidity, always close foil bag when packs are in use.

Lot number / Expiry date: The lot number and the expiry date are indicates on each Biolon packaging. In case of claims, please always indicate the lot number of the product. Do not use the product after the expiry date.

The splints produced from Biolon can be built up with dental acrylic resins.

Possible dangers

Before use observe the Material Safety Data Sheet of Biolon.

Storage life

2 years in closed original packaging.

Delivery forms

Ø 120 mm, clear transparent
(special measures can be delivered on request)

REF	D420001	0.5 mm	25 pcs.
REF	D420002	0.5 mm	50 pcs.

REF	D420019	0.625 mm	20 pcs.
REF	D420020	0.625 mm	40 pcs.

REF	D420003	0.75 mm	20 pcs.
REF	D420004	0.75 mm	40 pcs.

REF	D420005	1 mm	25 pcs.
REF	D420006	1 mm	50 pcs.

REF	D420007	1.5 mm	30 pcs.
REF	D420008	1.5 mm	60 pcs.

REF	D420009	2 mm	20 pcs.
REF	D420010	2 mm	60 pcs.

REF	D420011	3 mm	10 pcs.
REF	D420012	3 mm	50 pcs.

This product was developed for dental use and has to be used according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability and possible use in self dependence before using it subject to its purpose, especially in case the purpose is not mentioned in the instructions.

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