SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

EC REP

SHOFU DENTAL GmbH
An der Pönt 70, 40885 Ratingen, Germany

05.0:2020-09
300030334

Brownie Greenie Supergreenie

en

READ CAREFULLY BEFORE USE

PRODUCT DESCRIPTION

Brownie, Greenie, Supergreenie silicone polishers are for polishing dental restorations. Brownies are for prepolishing, Greenies are for polishing and Supergreens (with yellow banded shank) are for superpolishing.

PRECAUTIONS AND WARNINGS

- Never exceed the maximum permissible rotation speeds described in Table A.
- Use FG type with lower speed airmotor handpiece. High speed air turbine units can break cups or points.
- Mount an unmounted wheel onto the mandrel securely.
- Insert the shank completely into the chuck of the handpiece.
- Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Avoid any use at acute angles or with excessive pressure.
- Always use an unmounted wheel in clockwise motion.
- Use local dust extractor and protective equipment while using.
- Amalgam should be polished under running water to avoid heat generation. Thoroughly remove amalgam dust.
- This product is intended for use by dental professionals only.
- Do not use this product for any purposes other than the intended use.
- Dispose in accordance with relevant local and national regulation.

STORAGE AND STERILIZATION

- Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination.
- Perform autoclave under the condition described in Table B.
- When using intraorally, sterilize before each use.
- For reuse after using, remove any contaminants immediately with washer disinfectant, then sterilize.
- Follow the instructions for use of washer disinfectant and sterilizer, respectively.
- Do not use chlorine disinfectant liquid.
- For the details about REPROCESSING procedures, visit www.shofu.de.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

Location of the symbols glossary:
https://www.shofu.com/en/symbols_glossary/

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MAXIMUM PERMISSIBLE ROTATION SPEED (Table A)

Cup FG, Floppie	15,000 min ⁻¹
Unmounted Wheel, Disk, Supergreenie PC2	20,000 min ⁻¹
Others	30,000 min ⁻¹

en MAXIMUM PERMISSIBLE ROTATION SPEED
de MAXIMAL ZULÄSSIGE DREHZAH
fr VITESSE MAXIMALE DE ROTATION AUTORISÉE
nl MAXIMALE TOELAATBARE ROTATIESNELHEID
es VELOCIDAD MÁXIMA PERMISIDA DE ROTACIÓN
it MASSIMA VELOCITÀ DI ROTAZIONE CONSENTITA
sv HÖGSTA TILLÅTNA ROTATIONSASTIGHET
pt VELOCIDADE DE ROTAÇÃO MÁXIMA ADMISSÍVEL

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