Cavicide™

Disinfectant solution for surfaces of non-invasive medical devices

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Kerr

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* If used as a disinfectant spray,
the contact time is 1 minute.
If used as an immersion
disinfectant solution, the
contact time is 3 minutes.

Use-by date } SEE CONTAINER
 LOT

ORGANISM

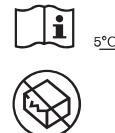
<i>Mycobacterium tuberculosis</i> var: bovis (BCG)	3
<i>Methicillin Resistant Staphylococcus aureus</i> (MRSA)	3
<i>Pseudomonas aeruginosa</i>	1
<i>Salmonella enterica</i>	3
<i>Staphylococcus aureus</i>	1
<i>Vancomycin Resistant Enterococcus faecalis</i> (VRE)	2
<i>Staphylococcus aureus</i> with reduced susceptibility to vancomycin	2
<i>Enterococcus hirae</i>	1
<i>Escherichia coli</i>	1
<i>Proteus mirabilis</i>	1
<i>Human Coronaviruses</i>	2
<i>Hepatitis B Virus</i> (HBV)	2
<i>Hepatitis C Virus</i> (HCV)	2
<i>Human Immunodeficiency Virus</i> (HIV-1)	2
<i>Herpes Simplex Virus Types 1 and 2</i>	2
<i>Influenza A2 Virus</i>	2
<i>Vaccinia Virus Ankara</i> (MVA)	5
<i>Candida albicans</i>	1*
<i>Trichophyton mentagrophytes</i>	3

*CONTACT TIME
(MINUTES)

CE
0044

Rx only

2673207r17
2020-10-16

SpofaDental a.s.
Markova 238,
506 01 Jíčín
Czech Republic,
www.spofadental.com
+420 493 583 111
Made in Czech Republic

Metrex
INFECTION PREVENTION

CaviCide™

(en)

Disinfectant solution for surfaces of non-invasive medical devices. Tuberculocidal, Bactericidal, Virucidal, Fungicidal, and Yeasticidal.

Directions for use: For professional use only. Do not use if seal is broken. See container for lot number and expiration date.

For use as a disinfectant on non-invasive medical device surfaces: Spray CaviCide directly onto pre-cleaned medical device surfaces, thoroughly wetting area to be disinfected. Allow medical device surfaces to remain visibly wet for required contact time at room temperature. Wipe surface dry using a towel or allow surface to remain undisturbed until visibly dry.

For use as a disinfectant soak for pre-cleaned, non-invasive medical devices and implements: Medical devices must be thoroughly cleaned to remove excess organic debris, rinsed and dried. Thoroughly clean and rinse lumens of hollow medical devices. Using either a tray or an ultrasonic unit, immerse medical devices in CaviCide for required contact time at room temperature. Remove and rinse medical devices. Wipe dry prior to use. Discard solution after each use.

Storage and disposal: Store in a well-ventilated place. Keep cool. Dispose of contents/container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation. Discard 90 days after opening.

Warning: Causes serious eye irritation. Keep away from heat, hot surfaces, open flames, sparks. No smoking. Keep container tightly closed. Wear eye protection, protective gloves. If eye irritation persists: Get medical advice/attention.

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