## Citric Acid 20% Solution

### Indications for use:

Citric Acid 20%, a root canal cleanser, is intended to etch root canal walls just prior to obturation to allow an optimum seal.

### Description:

Citric Acid 20% is a slightly thickened, water-soluble solution. It is tinted slightly orange for easy identification and is provided in a 30ml IndiSpense® Syringe.

### Preparation:

1. An empty 1.2ml or 5ml syringe is attached to the end of the large, no-waste IndiSpense syringe by turning the luer lock of the unit dose syringe snugly onto the male thread of the IndiSpense as show in Fig. 1.
2. With palm grasp (Fig. 2) the plunger of the IndiSpense syringe is depressed while the unit dose syringe is stabilized at the plunger with the nondominant hand. Slightly pull out on plunger of unit dose syringe to facilitate loading. Do not express contents of unit dose syringe back into IndiSpense syringe. To prevent cross-contamination, a used syringe should never be attached to the IndiSpense syringe.
3. Remove luer lock cap from syringe. Twist a NaviFX® Tip or NaviTip® firmly onto syringe. Gauge NaviFX Tip with the rubber stop at least 2mm short of the root terminus as show in Fig. 3.

### Procedure:

**Warning:** Do not follow Citric Acid rinse with sodium hypochlorite or vice versa, as this can lead to the production of chlorine gas. Always rinse with clean, bacteria-free water between Citric Acid and sodium hypochlorite irrigation.

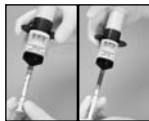
1. Make sure NaviFX Tip is loose in canal and at least 2mm short of terminus. Gently deliver the Citric Acid with NaviFX Tip. Move brush tip up and down to agitate and brush solution, cleaning walls to dissolve and remove calcium hydroxide with Citric Acid solution as shown in Fig. 4. Do not express Citric Acid when canula tip is at full length or if it fits tightly against the canal walls.
2. Flush out canal with clean, bacteria-free water, followed by copious sodium hypochlorite (e.g. ChlorCid®). When Citric Acid is used in lieu of EDTA liquid to remove smear layer, let Citric Acid dwell in canal for 3 minutes. Perform a final rinse with clean, bacteria-free water, plain EDTA or local anesthetic. Quick and effective removal of most of the Citric Acid solution or other irrigants is facilitated by using a Luer Vacuum Adapter with a Capillary Tip or 10mm Micro Capillary Tip.
3. One or two paper points are usually all that are required for final drying of canals if canals are first vacuumed out as described in step 2 above.
4. Fit master cone, then obturate. Ultradent recommends the EndoREZ® Obturation System including EndoREZ root canal sealer, EndoREZ resin-coated gutta percha points, and specially designed delivery systems.

### Warnings:

Do not follow Citric Acid rinse with sodium hypochlorite or vice versa, as this can lead to the production of chlorine gas. Always rinse with clean, bacteria-free water between Citric Acid and sodium hypochlorite irrigation.

### Precautions:


1. Always use a rubber dam for endodontic therapy. Prevent ingestion of Citric Acid solution.
2. Prevent getting solution in the eyes or leaving it on the soft tissues for an extended period of time.
3. Do not extend tip past the root apex or express Citric Acid past the apex.
4. Do not express Citric Acid from the canula tip at the root terminus or if the tip fits tightly against the canal walls. This may force solution past the apex.
5. Do not use this product on patients with a known sensitivity to Citric Acid.
6. The tips and delivery syringes are disposable and intended for single use.

*Figs. 1&2 A 1.2ml unit dose syringe is twisted firmly onto the IndiSpense syringe. While supporting plunger of unit dose syringe with the non-dominant hand, depress plunger of IndiSpense and fill unit dose syringe to desired level.*

*Fig. 3 Measure NaviFX tip 2mm short of root apex and loosely fitted in canal.*

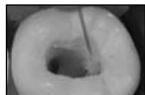*Fig. 4 Express Citric Acid into canal. Scrub tip up and down. Rinse canal thoroughly with plain EDTA, clean bacteria free water or local anesthetic.*

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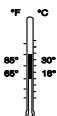
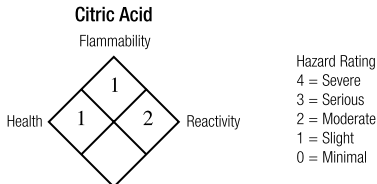
Ultradent syringes have an expiration date stamped on the side of the syringe consisting of one letter and three numbers.

The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date.

The first two numbers are the month, and the third number is the last number of the year.

**For product MSDS please see our  
website: [www.ultradent.com](http://www.ultradent.com)**

---

Store at room temperature.  
For Professional Use Only.

For immediate reorder and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free **1-800-552-5512**. Outside U.S. call (801) 572-4200 or visit our website at [www.ultradent.com](http://www.ultradent.com).

© Copyright 2007 Ultradent Products, Inc. All Rights Reserved.  
U.S. Patent Nos.: 5,697,918.

505 West 10200 South  
South Jordan, Utah 84095, USA

**U<sub>2</sub> ULTRADENT**  
PRODUCTS, INC.

#69720.2 053008