CE

en

READ CAREFULLY BEFORE USE

DirectDia is a diamond impregnated polishing paste used both intra and extra orally for achieving a high luster polish on both the restoration and natural tooth surface.

INDICATIONS

- High luster polishing of restorative materials (porcelain, direct composite, lab composite, metals etc.)
- Polishing and cleaning of natural teeth

DIRECTIONS FOR USE

- Remove cap from syringe and attach the delivery tip.
- Apply a small quantity of paste to the Super-Snap Buff Disk (sold separately). Do not apply this material directly to the teeth.
- A dry surface is not required. Use on a saliva moist surface is acceptable.
- Rinse thoroughly with water to remove all paste.

CAUTIONS AND RECOMMENDATIONS

- Finish the surface with silicone polisher before using.
- Attach the delivery tip to syringe securely and tightly.
- Use with Super-Snap Buff Disks – standard or mini (REF L523 or L524).
- To prevent splatter and avoid excessive heat generation always use rotary instruments at speeds below 10,000 min⁻¹.
- The quality of the product is not affected if liquid separation and pale color by liquid separation occurs. If the paste hardens it can be adjusted by adding water.
- Store in a dark place away from direct sunlight.
- Store at temperatures between 1°C/34°F and 30°C/50 °F.
- Replace cap tightly after use to prevent drying.
- Wearing protective glasses is recommended for patient and operator.
- Do not use this product on patients with known allergies to this material and/or its components.
- Operators with known allergies to this material and/or its components must not use this product.
- If any inflammation or other allergic reactions occur on either the patient or operator, immediately discontinue use and seek medical advice.
- In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

Location of the symbols glossary: https://www.shofu.com/en/symbols_glossary/

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SHOFU DENTAL ASIA-PACIFIC PTE.LTD. 10 Science Park Road, #03-12, The Alpha, Science ParkII, Singapore 117684
EC REP | SHOFU DENTAL GmbH An der Pönt 70, 40885 Ratingen,Germany