Dura-White Stones

Instructions for use

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Mode d'emploi

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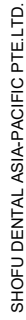
SHOFU INC.
11 Kamiakamatsu-cho, Fukuine,
Higashiyama-Ku, Kyoto 605-0983, Japan
www.shofu.com

0123

SHOFU DENTAL GmbH
Ander Pont 70, 40885 Ratingen, Germany

10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

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READ CAREFULLY BEFORE USE

PRODUCT DESCRIPTION

Made of aluminum oxide grits with ceramic binder, Dura-White Stones are ideal for fine contouring and finishing of enamel, composites and porcelains. They are available in friction grip (FG), contra-angle (CA) and hand piece (HP).

INTENDED PURPOSE

Fine contouring and finishing of enamel, composites, and porcelains

PRECAUTIONS AND WARNINGS

- Never exceed the maximum permissible rotating speed.
30,000 min⁻¹
- Insert the shank completely into the chuck of the handpiece, following the manufacturer's instructions.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Make sure that abrasive head is in good condition before use. Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt, exposure of shank due to repeated use.
- Use local dust extractor, protective mask, etc while contouring and finishing to avoid adverse effect of dust to the human body.
- Use protective equipment such as glasses while using.
- Avoid any use at acute angles or with excessive pressure to thin and long abrasives. They are fragile and may bend or fracture under excessive loads.
- Apply gentle pressure under water spray to avoid damaging pulp.
- This product is intended for use by dental professionals only.
- Do not use this product for any purposes other than specifically outlined in these instructions for use.

STORAGE AND STERILIZATION

Store this product avoiding exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination. When using intraorally, sterilize this product before each use. For reuse after using, remove any contaminants immediately with washer disinfectant, then sterilize.

(Autoclave)

Cycle Type	Temperature	Holding time	Dry time
Pre-vacuum 3 times	134°C 273°F	3'00"	10'00"

Follow the instructions for use of washer disinfectant and sterilizer, respectively. Chlorine disinfectant must not be used to avoid corrosion of the instruments. For the details about REPROCESSING procedures, visit www.shofu.de.

DISPOSAL

Dispose in accordance with relevant local and national regulation.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

Location of the symbols glossary: https://www.shofu.com/en/symbols_glossary/

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