## IRRIGATION NEEDLE

Directions for use  
PD190805

Endodontic irrigating needles	Agulhas de irrigação endodônticas
Canules d'irrigation endodontique	Эндодонтические ирригационные иглы
Endodontische Spülkanülen	根管治疗冲洗针
Aghi endodontici per irrigazione	根管冲洗针头
Agujas de irrigación para endodoncia	

### FOR DENTAL USE ONLY

#### EN DIRECTIONS FOR USE ENDODONTIC NEEDLE FOR ROOT-CANAL IRRIGATION

Applicable for:  
EDDY® Flex.Cannula™ Irrigation Needle – Ref. MNSEDIRD2230G

- 1) INDICATION  
Root-canal irrigation.
- 2) PROPERTIES  
Flexible cannula with radial opening for liquid ejection along the canal walls.
- 3) INSTRUCTIONS FOR USE  
  - 1) Open the package by pulling along pre-cut notch provided for this purpose.
  - 2) Insert the "Secure Lock" part of the needle into the male nose of a previously filled Luer lock syringe and lock the needle by turning it.
  - 3) Make sure the needle is locked on the syringe.
  - 4) Use the syringe/needle assembly according to the usual protocol for canal irrigation.
  - 5) After use, disconnect the needle and dispose of it in accordance with local recycling regulations.
- 4) PRECAUTIONS  
Check the length of the canal to be irrigated and use the 18-, 19-, 20- and 22-mm length markings to ensure proper positioning of the needle. Make sure that the needle can move freely to ensure constant flowback of the irrigation liquid.
- 5) ADVERSE REACTIONS  
None known.
- 6) CONTRAINDICATIONS  
None known.
- 7) WARNING MESSAGES  
Single-use device. Do not reuse.
- 8) STORAGE  
Keep away from direct light.
- 9) DISINFECTION, CLEANING AND STERILISATION  
This product has been manufactured under clean-room conditions. It is ready for use. Do not sterilise before use.

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Distributed by: VDW GmbH