**EXAMIX™ NDS REGULAR TYPE**  
**EXAMIX™ NDS INJECTION TYPE**  
**EXAMIX™ NDS MONOPHASE**  
**EXAMIX™ NDS HEAVY BODY**
**HYDROPHILIC VINYL POLYSILOXANE IMPRESSION MATERIAL**

For use only by a dental professional in the recommended indications.  
**DESCRIPTION**  
 EXAMIX NDS is a significantly advanced addition reaction (platinum catalyzed) VPS impression material. It produces crisp, extraordinarily smooth and highly precise impressions for models, crowns, bridges and dentures.

**RECOMMENDED INDICATIONS AND USES**

Indication	Impression Technique	Tray Type	Recommended Viscosities
Impression for facing, inlay, onlay, crown or bridge	Single mix single impression		Monophase
	Double mix single impression	Stock or Custom Tray	Monophase or Heavy + Injection or Regular
	Double mix double impression	Putty + Injection or Regular	Monophase or Heavy + Injection or Regular
	Double mix single impression	Putty + Injection or Regular	Monophase or Heavy + Injection or Regular
	Putty-Wash impression		Putty + Injection or Regular
Impression of partial denture	Single mix single impression	Stock or Custom Tray	Monophase
	Putty-Wash impression	Stock Tray	Putty + Regular or Injection
Impression of full denture	Single mix single impression	Stock or Custom Tray	Monophase

**CONTRAINDICATIONS**

In rare cases, product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

**PHYSICAL PROPERTIES (Average)**

Test Items	Injection Type 3: Light-Bodied Consistency	Regular Type 2: Medium-Bodied Consistency	Monophase Type 2: Medium-Bodied Consistency	Heavy Type 1: Heavy-Bodied Consistency
Total Working Time (Min., Sec.)	2'15"	2'00"	2'00"	2'00"
Minimum Time in Mouth (Min., Sec.)	4'00"	4'00"	4'00"	4'00"
Recovery From Deformation (%)	99.7	99.7	99.5	99.5
Maximum Strain in Compression (%)	≤7.0	≤6.0	≤3.3	≤3.3
Linear Dimensional Change after 24 hrs (%)	≤0.2	≤0.2	≤0.2	≤0.2

Test conditions: Temperature (74°F/23°C ± 4°F/2°C) Relative humidity (50 ± 5%) (ISO 4823:1992(E))

**DIRECTIONS FOR USE**


Note: The cartridge and mixing tip provided are compatible with GC Cartridge Dispenser 2.

**CARTRIDGE LOADING AND DISPENSING**

- Lift the release lever of the CARTRIDGE DISPENSER 2. (referred to as the dispenser hereafter) and pull the piston plunger all the way back into the dispenser. Lift the cartridge holder of the dispenser and load the cartridge, ensuring that the V-shaped notch on the flange of the cartridge is facing down. Push the cartridge holder down to hold the cartridge firmly in place.
- Lift the release lever and push the piston plunger forward until it engages into the cartridge.
- Remove the cartridge cap by rotating 1/4 turn anti-clockwise. Tilt the cap downward and peel it away from the cartridge. Gently squeeze the dispenser handle to extrude a small amount of material from the two openings at the end of cartridge. Make sure that base and catalyst come out evenly.
- Align the V-shaped notch on the rim of mixing tip to the V-shaped notch between the cartridge barrels. Push firmly to attach the mixing tip. Then rotate the colored collar of the mixing tip 1/4 turn clockwise to the end of the cartridge. The dispenser is now ready for use.
- Squeeze the handle several times to extrude the material. After use, do not remove the mixing tip as this will become the storage cap until next use. When replacing the mixing tip, rotate the collar on the mixing tip 1/4 turn anti-clockwise to align the V-shaped notch on the cartridge. Tilt the mixing tip downward and peel it away from the cartridge.
- Remove and replace the old mixing tip immediately prior to next use. Before attaching a new tip, gently extrude a small amount of material to ensure that base and catalyst are flowing evenly from both openings. If the materials should fail to extrude, remove any hardened materials from the end of the cartridge.
- To replace the cartridge, lift the release lever and retract the piston plunger fully. Remove the empty cartridge by lifting the cartridge holder and load a new one into the dispenser.

**USING THE IMPRESSION MATERIAL**

- Note: Prior to use, leave the material to stand at room temperature. Cooled or refrigerated material may delay setting.
- Attach an intraoral tip to the mixing tip or load material into a syringe.
  - Inject the mixture onto the prepared teeth.
  - Load the tray with the appropriate material (putty, monophase, heavy body or regular - depending on technique being used).
  - Seat the loaded tray in the mouth within specified working times.
  - Wait for set (4 minutes in the mouth).
  - The obtained impression should be cleaned, then disinfected, utilizing a 2.5% or 3.4% glutaraldehyde, or other appropriate disinfectant, according to the manufacturers label recommendations.
  - Remove the impression and pour a model immediately (if desired). Maximum time for pouring the model is 14 days.

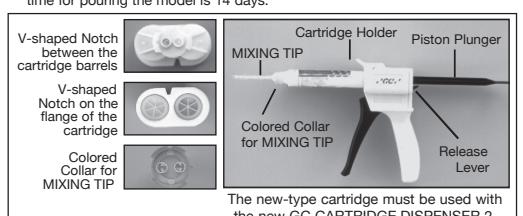The new-type cartridge must be used with the new GC CARTRIDGE DISPENSER 2.

**NOTES**

- When extruding EXAMIX NDS, care should be taken to avoid mixing or contact with the following materials. They may delay or prevent setting:
  - Catalyst for condensation type silicone impression materials
  - Polyulfide impression materials
  - Eugenol materials
  - Sulfur
  - Acrylates
 Also avoid moisture and glycerol when extruding.

- Avoid use of EXAMIX NDS with patients who have a history of hypersensitivity to silicone impression materials.

**STORAGE**

Recommended for optimal performance, store at room temperature (74°F/23°C) ± 50% relative humidity.

**PACKAGES**

Note: All packaging types may not be available in all countries.

**EXAMIX NDS 2 Cartridge Package with Tips**

- Regular
- Monophase
- Injection
- Heavy

**EXAMIX NDS 32 Cartridge Package Refill without Tips**

- Regular
- Monophase
- Injection
- Heavy

**EXAMIX NDS 80 Cartridge Package Refill without Tips**

- Regular
- Monophase
- Injection
- Heavy

**CAUTION**

- Avoid contact of base, catalyst, mixed material or adhesive with eyes. In case of contact, immediately flush with generous amount of water and seek medical attention.
- Universal VPS Adhesive (optional) is flammable. Do not use or store near heat or open flame. Use in a well ventilated area.
- Avoid getting material on clothing as it is hard to remove.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

**CLEANING AND DISINFECTION**

- MULTI-USAGE SYSTEM: To avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.

DO NOT IMMERSER. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

<http://www.gc-europe.com>

They can also be obtained from your supplier.

**UNDESIRED EFFECTS - REPORTING:**

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

[https://ec.europa.eu/growth/sectors/medical/devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical/devices/contacts_en)

as well as to our internal vigilance system:

<http://gcmc.dental>

In this way we will contribute to improve the safety of this product.

Last revised: 02/2021

Rx Only

CE

MANUFACTURED by  
GC CORPORATION

7-11-12-13, Itabashi-ku, Tokyo 174-8585, Japan

Tel: +81-3-3965-1221

RESPONSIBLE MANUFACTURER IN CANADA:  
GC AMERICA INC.

3737 West 127th Street, Alsip IL 60803 U.S.A.

DISTRIBUTED by  
GC AMERICA INC.

3737 West 127th Street, Alsip IL 60803 U.S.A.

Tel: +1-708-597-9900

[www.gcamerica.com](http://gcamerica.com)

EU: GC EUROPE N.V.

Havenstraat 10, B-1000 Hasselt-Leuven 1240, Interleuvenlaan 33, B-3001 Leuven, Belgium

Tel: +32 16 74 10 00

GC ASIA DENTAL PTE. LTD.

5 Tampines Central 1, #06-01, Tampines Plaza 2, Singapore 529541

Tel: +65 6546 7588

GC AUSTRALASIA DENTAL PTY. LTD.

153 Old Kingsmeadow, NSW 2019, Australia

Tel: +61 2 9301 8200

GC SOUTH AMERICA

Rua Heliodoro, 399, Santana - São Paulo, SP, BRASIL

CEP: 02022-051 - TEL: +55-11-2925-0965

CNPJ: 03.279.995/0001-61 RESP: TEC. Erick de Lima - CRO/SP 100.866

3020221

**EXAMIX™ NDS REGULAR TYPE**  
**EXAMIX™ NDS INJECTION TYPE**  
**EXAMIX™ NDS MONOPHASE**  
**EXAMIX™ NDS HEAVY BODY**
**HYDROPHILES VINYL-POLYSILOXAN ABFORMMATERIAL**

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**CONTRAINDICATIONS**

In rare cases, product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

**PHYSICAL PROPERTIES (Average)**

Test conditions: Temperature (74°F/23°C ± 4°F/2°C) Relative humidity (50 ± 5%) (ISO 4823:1992(E))

**DIRECTIONS FOR USE**

Note: The cartridge and mixing tip provided are compatible with GC Cartridge Dispenser 2.

**CARTRIDGE LOADING AND DISPENSING**

1. Lift the release lever of the CARTRIDGE DISPENSER 2. (referred to as the dispenser hereafter) and pull the piston plunger all the way back into the dispenser. Lift the cartridge holder of the dispenser and load the cartridge, ensuring that the V-shaped notch on the flange of the cartridge is facing down. Push the cartridge holder down to hold the cartridge firmly in place.

2. Lift the release lever and push the piston plunger forward until it engages into the cartridge.

3. Remove the cartridge cap by rotating 1/4 turn anti-clockwise. Tilt the cap downward and peel it away from the cartridge. Gently squeeze the dispenser handle to extrude a small amount of material from the two openings at the end of cartridge. Make sure that base and catalyst come out evenly.

4. Align the V-shaped notch on the rim of mixing tip to the V-shaped notch between the cartridge barrels. Push firmly to attach the mixing tip. Then rotate the colored collar of the mixing tip 1/4 turn clockwise to the end of the cartridge.

5. Squeeze the handle several times to extrude the material. After use, do not remove the mixing tip as this will become the storage cap until next use. When replacing the mixing tip, rotate the collar on the mixing tip 1/4 turn anti-clockwise to align the V-shaped notch on the cartridge. Tilt the mixing tip downward and peel it away from the cartridge.

**EXAMIX™ NDS REGULAR TYPE**  
**EXAMIX™ NDS INJECTION TYPE**  
**EXAMIX™ NDS MONOPHASE**  
**EXAMIX™ NDS HEAVY BODY**
**HYDROPHILIC VINYL POLYSILOXANE IMPRESSION MATERIAL**

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