
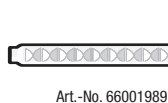
## Cartridge Delivery

- Bite registration Tip
- Mixing Tip
- Dispensing Gun 2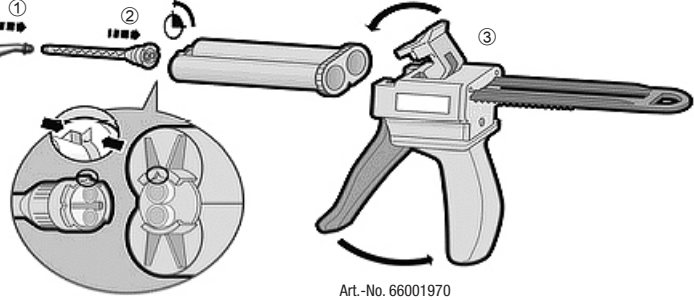

Distributed in USA/Canada exclusively by:  
Kulzer LLC  
300 Hereaus Way  
South Bend, IN 46614-2517  
1-800-431-1785

Manufacturer:  
Kulzer GmbH  
Leipziger Straße 2  
63450 Hanau (Germany)  
Made in Romania

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Status: 2019-08

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### Scannable polyvinyl siloxane for bite registration, superior hardness, very short time in mouth.

**Description**  
Flexitime Bite is an addition-cross-linking polyvinyl siloxane with a very high final Shore D hardness of 40. Its surface properties and optimized color enable it to be optically recorded in CAD/CAM systems to portray the opposing dentition. Flexitime Bite can be scanned in the Kulzer scanner and many other CAD/CAM systems. It complies with the specifications in DIN 13903.

#### Uses

- Standard bite registrations in the end bite position
- Portrayal of opposing dentition
- Coating of bite forks
- Key material for needle point registration
- Production of small model segments

#### Contraindications

Flexitime Bite should not be used for conventional impressions since its high final hardness makes it very difficult to remove from the mouth. Undesired tooth extractions may result. The use of this product is contraindicated in case of known or suspected allergies against components of this product.

#### Side effects

This product or one of its components may in particular cases cause hypersensitive reactions. If suspected, information on the ingredients can be obtained from the manufacturer.

#### Instructions for use

Please visually inspect for damage before use. Do not use damaged products. Mixing tips and bite registration tips are for single use only. To avoid cross contamination Bite Registration Tips and Mixing Tips must be disposed of immediately after initial use.

Dispense a small amount of material from a 50 ml cartridge before attaching a mixing tip. After use, remove the mixing tip and keep it closed. Immediately before the next use, remove the mixing tip and attach a new one.

When bite registration materials based on polyvinyl siloxane are touched with latex gloves, retraction solutions or freshly inserted composite fillings, it can change the curing behavior. Flexitime Bite should be at room temperature when used (23°C (73°F)). Lower temperatures, for example when the materials are stored in a refrigerator can extend the working time, and higher temperatures can shorten it.

Flexitime Bite is used with Mixing Tip green (1:1). It can also be combined with the Bite Registration Tip.

## Instructions for use GB

### Scannable polyvinyl siloxane for bite registration, superior hardness, very short time in mouth.

**Description**  
Flexitime Bite is an addition-cross-linking polyvinyl siloxane with a very high final Shore D hardness of 40. Its surface properties and optimized color enable it to be optically recorded in CAD/CAM systems to portray the opposing dentition. Flexitime Bite can be scanned in the Kulzer scanner and many other CAD/CAM systems. It complies with the specifications in DIN 13903.

#### Uses

- Standard bite registrations in the end bite position
- Portrayal of opposing dentition
- Coating of bite forks
- Key material for needle point registration
- Production of small model segments

#### Contraindications

Flexitime Bite should not be used for conventional impressions since its high final hardness makes it very difficult to remove from the mouth. Undesired tooth extractions may result. The use of this product is contraindicated in case of known or suspected allergies against components of this product.

#### Side effects

This product or one of its components may in particular cases cause hypersensitive reactions. If suspected, information on the ingredients can be obtained from the manufacturer.

#### Instructions for use

Please visually inspect for damage before use. Do not use damaged products. Mixing tips and bite registration tips are for single use only. To avoid cross contamination Bite Registration Tips and Mixing Tips must be disposed of immediately after initial use.

Dispense a small amount of material from a 50 ml cartridge before attaching a mixing tip. After use, remove the mixing tip and keep it closed. Immediately before the next use, remove the mixing tip and attach a new one.

When bite registration materials based on polyvinyl siloxane are touched with latex gloves, retraction solutions or freshly inserted composite fillings, it can change the curing behavior. Flexitime Bite should be at room temperature when used (23°C (73°F)). Lower temperatures, for example when the materials are stored in a refrigerator can extend the working time, and higher temperatures can shorten it.

Flexitime Bite is used with Mixing Tip green (1:1). It can also be combined with the Bite Registration Tip.

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\* *Gelson Luis Adabo, DDS, PhD, Elaine Zanoarotti, Renata Garcia Fonseca, and Carlos Alberto dos Santos Cruz, DDS, PhD – Effect of disinfectant agents on dimensional stability of elastomeric impression materials. J Prosthet Dent, 81, 621–4, 1999*

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\*1 Total working time (including mixing) at 23°C (73°F); shortened to 20s at oral temperature (35°C (95°F))

\*2 Minimum time that the bite registration material must be in a closed mouth. The data given are furnished to the best of our knowledge and are based on internal test results.

#### Disinfection

Flexitime Bite can be treated with an aqueous sodium hypochlorite solution (5,25% for up to 10 minutes\*). The treatment does not affect the surface or dimensions.

\* *Gelson Luis Adabo, DDS, PhD, Elaine Zanoarotti, Renata Garcia Fonseca, and Carlos Alberto dos Santos Cruz, DDS, PhD – Effect of disinfectant agents on dimensional stability of elastomeric impression materials. J Prosthet Dent, 81, 621–4, 1999</*

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