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Deutsch - FFS32XX-04/21

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FANTESTIC® FLOWFILL

CE 0482

### Hersteller:

R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg  
T 040-30707073-0 • F 040-30707073-73 • I www.r-dental.com

### FANTESTIC® FLOWFILL

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### FANTESTIC® FLOWFILL

REF FFS32XX (in syringes, XX depends on shade)

#### Description

FANTESTIC® FLOWFILL is a light-cure and flowable microhybrid composite for anterior and limited posterior restorations and for sealings.

#### Characteristics and composition

FANTESTIC® FLOWFILL is based on methacrylate-resin and inorganic filler particles of a size of 0.05-1.0 µm. The total filler load is 62 wt.-% and 47 vol.-%.

The radiopacity of FANTESTIC® FLOWFILL is 150 % aluminium (radiopacity of enamel: ~ 200 % aluminium, of dentin: ~ 100 % aluminium)

FANTESTIC® FLOWFILL-products are available in VITA®-shades.

FANTESTIC® FLOWFILL meets the requirements of DIN EN ISO 4049, type 1, class 2, group 1.

#### Indications

FANTESTIC® FLOWFILL is indicated for

- anterior restorations (class III, IV),
- restorations of class V (wedge-shaped defects, cervical caries),
- small posterior restorations (class I, II),
- as a liner in cavities (initial layer in cavities) and
- extended pit & fissure sealings.

#### Side effects

Irritations resulting from direct contact with the pulp are possible. Therefore for pulp protection areas close to the pulp must be covered with a thin layer of calcium hydroxide or a bioactive pulp capping material, see section 4.

#### Incompatibility with other materials

Do not use FANTESTIC® FLOWFILL in combination with eugenol containing materials. Eugenol impairs the polymerization of the composite. Do not store FANTESTIC® FLOWFILL in proximity of eugenol containing materials. Unpolymerized FANTESTIC® FLOWFILL should not come in contact with eugenol containing materials.

#### Recommended procedures

##### 1. Preparation

Prior to the preparation a cleaning of the tooth with a fluoride-free paste (e. g. flour of pumice with water) is recommended.

##### 2. Shade matching

Ascertain the tooth shade while teeth are still moist and select the appropriate FANTESTIC® FLOWFILL shade (FANTESTIC® FLOWFILL-products are available in VITA®-shades).

##### 3. Isolation

A dry area and use of rubber dam (REF KDL31XX, XX = depends on strength) or latex-free dam (REF KDL3500) is recommended.

##### 4. Preparation of the cavity

The cavity will then be prepared in accordance with clinical rules and with minimal tooth reduction.

For pulp protection areas close to the pulp should be protected with a thin layer of calcium hydroxide, Cal de Luxe®, (Ca(OH)<sub>2</sub>, "ready-to-use" in a portion blister, REF CDL2503) according to the instructions for use or with a pulp cap material, a bioactive pulp capping material, like ReViCal (REF RVC1002) according to the instructions for use.

Then apply the Total-Etch-Technique (Etch & Rinse) or the Self-Etch-Technique with common adhesives (like FANTESTIC® UniversalBond (REF FUB6000) as a self-etching one-component adhesive) for dentin and enamel according to the instructions for use.

Then FANTESTIC® FLOWFILL can be applied directly.

##### 5. Application of the flowable composite FANTESTIC® FLOWFILL

FANTESTIC® FLOWFILL will then be applied for an optimal result in thin layers of max. 1 mm into the cavity. Transparent matrices are recommended.

After use waste the disposable application cannula (REF FFK3281) and screw on the original cap to reseal the syringe.

The application cannulas are for single use only.

##### 6. Light-cure

FANTESTIC® FLOWFILL can be light-cured with commercial polymerization units (halogen or LED) with a wave length of 400-500 nm and with a light intensity > 1000mW/cm<sup>2</sup>.

Cure each layer of lighter shades (e. g. A1, A2, C2) **20 s** and of darker shades (e. g. A3,5, B3) **30 s**. Hold the light emission window as close as possible to the filling material.

Attention: The ambient light of the dental lamp may start polymerization of the composite.

##### 7. Finishing and high gloss polishing

After removing the matrix FANTESTIC® FLOWFILL can be contoured with finishing diamonds, flexible grinding discs or carbide burs. Polish to high gloss with polishing discs or polishing brushes.

##### 8. Storage

Do not store above 77 °F! Avoid exposure to sunlight. Do not use after expiration date. Close syringe immediately after use to avoid exposure to light.

##### Precautions to application, product life and/or storage

Avoid contact of the etching gel FANTESTIC® ETCH, calcium hydroxide Cal de Luxe® and the pulp capping material ReViCal with soft tissues, skin or eyes. If there is an accidental contact, flush immediately with copious amounts of water.

Pay attention to the Material Safety Data Sheets (MSDS >> www.r-dental.com).

The ambient light of the dental lamp may start polymerization of the composite.

Do not adjust the viscosity of the composite restorative material by using any resin.

The contact of FANTESTIC® FLOWFILL with skin should be avoided, especially by anyone having known resin allergies against methacrylates.

Commercial medical gloves do not protect against methacrylates.

Shelf life see label. Do not use after expiry date.

Do not store above 77 °F (25 °C)! Avoid exposure to sunlight.

\*VITA is not a registered trademark of R-dental Dentalerzeugnisse GmbH.

FANTESTIC and Cal de Luxe are registered trademarks of R-dental Dentalerzeugnisse GmbH.

Keep out of the reach of children!

**Caution:** For dental use only.

##### Warranty

R-dental Dentalerzeugnisse GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalerzeugnisse GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and R-dental Dentalerzeugnisse GmbH's sole obligation shall be repair or replacement of the product.

##### Limitation of Liability

Except where prohibited by law, R-dental Dentalerzeugnisse GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2021 R-dental Dentalerzeugnisse GmbH. All rights reserved.