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#### EN: Instructions for use

Intended use: Thermoforming sheets for the manufacture of apparatus for dental and orthodontic treatment.

Field of application: Foliflex is thermoformed to fabricate intra-oral appliances such as positioners, bracket transfer splints, soft splints, radiation protection splints, Playsafe sports mouthguards.

Contraindications: In case of already known allergic reaction to plastics a specific allergy to Foliflex (EVA) should be excluded.

Precautionary measures: Foliflex is only recommended for the above mentioned devices. Observe storage instructions.

Attention: The use is limited to the production of dental and orthodontic appliances. Use is the responsibility of the dentist. Use only on the recommendation or prescription of an authorised dentist. For single use only (thermoforming). Allergic reactions are unlikely, but cannot be excluded. Incorrect processing can lead to cracks, fractures and sharp edges. There is a risk of aspiration in the case of small breakages.

Foliflex (approx. values): density 0,95 g/cm<sup>3</sup>, modulus of elasticity 20 MPa, elongation at break 850%, water absorption 0,2%, temperature resistance 54°C

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