Prior to use, carefully read the instructions for use.

EN

GC Fuji™ II CAPSULE

RADIOPAQUE GLASS IONOMER RESTORATIVE CEMENT IN CAPSULES

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

- Restoration of primary teeth.
- Core build-up.
- Restorations of Class III, V and limited Class I cavities.

CONTRAINDICATIONS

- Pulp capping
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.



DIRECTIONS FOR USE

Powder / Liquid Ratio (g/g)	0,30 / 0,11
Mixing Time (sec.)	10"
Working Time (min., sec.)	2'00"
Net Setting Time (min., sec.)	2'20"

Test conditions : Temperature (23 +/-1°C),
Relative humidity (50 +/-5%)

1. CAPSULE ACTIVATION AND MIXING

- Before activation, shake the capsule or tap its side on a hard surface to loosen the powder (Fig. 1).
- To activate the capsule, push the plunger until it is fully depressed with the main body and hold it down for 2 seconds (Fig.2).

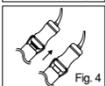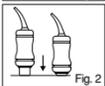- Note:
- Ensure the plunger is fully pressed to avoid the incorrect mixing ratio of powder and liquid.
 - The capsule should be activated just before mixing and used immediately.

2. RESTORATIVE TECHNIQUE

- Prepare tooth using standard techniques. Extensive mechanical retention is not necessary.
- Wash and dry but DO NOT DESICCATE. CAVITY CONDITIONER or DENTIN CONDITIONER is recommended to remove the smear layer. For pulp capping use calcium hydroxide.
- Mix the cement. Working time is 2 minutes from the start of mixing at 23°C (73.4°F). Higher temperatures will shorten working time.
- Immediately remove the mixed capsule from the mixer and load it into the GC Capsule Applier.
- Make two clicks to prime the capsule then syringe (Fig. 4).
- Remove surface moisture, but DO NOT DESICCATE.
- Extrude cement directly into preparation. Avoid air bubbles.

3. FINISHING

- One visit treatment
- After initial set, correct the contour WITHOUT WATER SPRAY using standard techniques.
- Apply GC Fuji VARNISH or GC Fuji COAT LC and wait for about 15 minutes.
- Final finishing and polishing under water spray can now be carried out.
- Apply a final coat of GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure for 10 sec.) or G-COAT PLUS (light cure for 20 sec.)
- Standard method
- 24 hours after placement finish and polish the restoration under water spray using standard techniques.

STORAGE

Recommended for optimal performance, store at temperature of 4-25°C (39.2-77.0°F).

SHADE

No. 21	pale yellow	No. 23	dark grey
No. 22	yellow brown	No. 23M	deep grey
No. 22.5	grey brown	No. 26	deep brown

PACKAGES

- Assortment package
50 capsules (5 pcs. No. 21, 26, 10 pcs. No. 22, 22.5, 23, 23M).
- Single shade package : 50 capsules
* Average contents per capsule : 0.30g powder and 0.11g (0.09mL) liquid.
* Minimum net volume of mixed cement per capsule : 0.12mL.
Option : GC Capsule Applier (1 piece).

CAUTION

- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS.
Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com>
They can also be obtained from your supplier.

Undesired effects- Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:
https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental
In this way you will contribute to improve the safety of this product.

Last revised : 06/2019

MANUFACTURED by
GC CORPORATION
76-1 Hasuruma-cho, Itabashi-ku, Tokyo 174-8585, Japan

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Prior to use, carefully read the instructions for use. **EN**

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CONTRAINDICATIONS

- Pulp capping.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

Powder / Liquid Ratio (g/g)	0.30 / 0.11
Mixing Time (sec.)	10"
Working time (min., sec.) (at 23°C)	
(from start of mixing)	2'00"
Net Setting Time (min., sec.)	2'20"

Test conditions : Temperature (23 ±1°C), Relative humidity (50 ±5%)

1. CAPSULE ACTIVATION AND MIXING

- Before activating, shake the capsule or tap on a hard surface to loosen the powder (Fig. 1).
- To activate the capsule push plunger until it is flush with the main body (Fig. 2).
- Immediately place the capsule into a metal GC Capsule Applier and click the lever once (Fig. 3). The capsule is now activated.

Note : The capsule should be activated just before mixing and used immediately.
- Immediately set the capsule into the mixer and mix for 10 seconds (+/- 4,000 RPM) (Fig. 4).
- RESTORATIVE TECHNIQUE**

Prepare the tooth using standard techniques. Extensive mechanical retention is not necessary.
- Wash and dry but **DO NOT DESICCATE**. GC CAVITY CONDITIONER or GC DENTIN CONDITIONER is recommended to remove the smear and prepare the surface for bonding with calcium hydroxide.
- Mix the cement. Working time is 2 minutes from the start of mixing at 23°C (73.4°F). Higher temperatures will shorten working time.
- Immediately remove the mixed capsule from the mixer and load it into the GC Capsule Applier.
- Make two clicks to prime the capsule then syringe (Fig. 5).
- Remove surface moisture, but **DO NOT DESICCATE**.
- Extrude cement directly into preparation. Avoid air bubbles.

Note : To adjust the direction of the nozzle, hold the applicer with the capsule towards you and turn the capsule body.
- To remove the used capsule, push the applicer release button. Twist the capsule and pull upwards.
- For the preliminary contour and cover with a matrix if required.
- Once set, remove the matrix and immediately apply GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure).

3. FINISHING

- One visit treatment
 - After initial set, correct the contour WITHOUT WATER SPRAY using standard techniques.
 - Apply GC Fuji VARNISH or GC Fuji COAT LC and wait for about 15 minutes.
 - Final finishing and polishing under water spray can now be carried out.
 - Apply a final coat of GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure for 10 sec.) or G-COAT PLUS (light cure for 20 sec.)
- Standard method

24 hours after placement finish and polish the restoration under water spray using standard techniques.

STORAGE

Store at temperature of 4-25°C (39.2-77.0°F). (Shelf life : 2 years from date of manufacture).

SHADE

No. 21 pale yellow
No. 22 yellow brown
No. 22.5 grey brown
No. 23 dark grey
No. 23M deep grey
No. 26 deep brown

PACKAGES

- Assortment package
 - 50 capsules (5 pcs. No. 21, 26, 10 pcs. No. 22, 22.5, 23, 23M).
- Single shade package : 50 capsules
 - * Average contents per capsule : 0.30g powder and 0.11g (0.09mL) liquid.
 - * Minimum net volume of mixed cement per capsule : 0.12mL.
 - Option : GC Capsule Applier (1 piece).

CAUTION

- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.

Last revised : 03/2014

CE 0086

'GC'

MANUFACTURED BY
GC CORPORATION
76-1 Hazumiya-cho, Itabashi-ku, Tokyo 174-8585, Japan

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