G-enial® Universal Flo

UNIVERSAL LIGHT-CURED RADIOPAQUE FLOWABLE COMPOSITE

For use only by a dental professional in the recommended application techniques.

RECOMMENDED INDICATIONS

- Direct restorative for Class I, IV, III, II and V Cavities.
- Fissure sealant
- Sealing hypersensitive areas
- Repair of direct aesthetic restorations, temporary crown, defect margins when margins are in enamel
- Sealing of orthodontic brackets
- Inner or base
- Realization of crowns & bridges, inlays and veneers
- Use of this product in combination with GC Grada components (please refer to its dedicated FU)
- Use of this product in combination with fibres such as GC EverStick fibres. Follow manufacturer's instructions.
- Adhesive luting of ceramic and composite inlays, crowns and veneers with a thickness < 2.0 mm that enables the use of a purely light-curing technique.

CONTRAINDICATIONS

- Direct pulp capping
- Use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.

Operation

- Preparations
 - Hold the curing upright and remove the cap by turning counterclockwise.
 - Place and securely attach the dispensing tip. The cap will be turned to the correct position.
 - Trim the light protective cover until use. Note: Take care not to attach the dispensing tip too tightly. This may damage its ring. In order to remove the cap, use a dental scaler that the threads are free of cement material.

Shade Selection


- Light protective cover
 - Clean the tooth with pumice and water. Shade selection should be made prior to isolation. Select the appropriate shade from 15 shades of A1, A2, A3, A3.5, A4, B1, B2, B3, C1, A02, A03, B0V, CV, JE, A.E. A, B, C, shades are based on the color of the tooth.
 - *A1, A2, A3, A3.5, A4, B1, B2, B3, C1 are registered trademarks of Vita Zahnfabrik, Bad Säckingen.

Bonding Treatment

- For pulp capping, use calcium hydroxide or dentine, use a light-cured bonding system such as Clearfil-Bond, GC G-BOND, Clearfil-Bond, Follow manufacturer's instructions.

Placement of G-enial Universal Flo

- Remove cover from the dispensing tip. Note: Prior to syringe material into the cavity, make sure to check the following:
 - Dispensing tip is securely attached to the syringe.
 - Any possible air for the dispensing tip is removed by gently pushing forward the syringe plunger with the tip pointing upwards until material comes out of the opening.

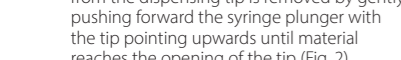Dispensing

- Place the dispensing tip as close as possible to the prepared cavity, and slowly push the plunger in a steady motion, allowing any dispensed material onto a mixing pad or transfer to the cavity using a suitable instrument.
- If plugged, remove the dispensing tip and extrude a small amount of material directly onto a mixing pad.
- Amberize exposure to ambient light.
- Amberize light can shorten the manipulation of the filling.
- After use, immediately remove and dispose the dispensing tip and tightly close the syringe with the cap.

Cinical Hint

- Inject effectively, use the shape/tenor of the material to ensure uniformity across the entire surface of the restoration during build up. Once the material is fully cured, use a rubber dam to ensure the pressure on the plunger and withdraw the syringe in a direction perpendicular to the surface. This will allow the material to separate from the surface and provide a smooth surface over the restoration.

Light Curing

- Light cure the G-enial Universal Flo using a light curing unit. Keep light curing as close as possible to the surface. Use a light curing unit with an irradiation time and effective depth of cure.

Irradiation Time and Effective Depth of Cure

Shade	Recommended Irradiation Time (sec)	Effective Depth (mm) (ISO 15187:2007)
A1, A2, A3, B1, B2, B3, C1, A02, A03, A04, A05	20	2.0

Note:

- Material should be placed and light cured in layers. For maximum layer thickness, please consult above table.
- Lower light intensity may cause insufficient curing or discoloration of the material.
- For the indirect technique light cure each layer thoroughly for 10 seconds with the GC STEELIGHT SL or for 10 seconds with GC STEELIGHT SL.

CEMENTATION OF INLAYS, ONLAYS AND VENEERS

- Preparation of the cavity
 - After removing the provisional restoration, clean the prepared tooth using a brush and slurry of pumice with water or a sponge soaked with alcohol. Flush with water to remove any slurry. Rinse the prepared tooth with water spray. Try-fit the restoration before the cementation and clean thoroughly.

Treatment of the restoration

- Make sure that the restoration is pretreated and handled according to the manufacturer's instructions.
- Apply Ceramic Primer II, G-Multi Primer or a similar silane coupling agent to the bonding surface of the restoration according to the manufacturer's instructions. Do not rinse it.

Treatment of tooth preparation

- Selectively etch with 35-37% phosphoric acid, rinse and gently dry thoroughly for 5 seconds using maximum air pressure. Refer to the respective instructions for use.
- Isolation with rubber dam is recommended.
- Make sure to blow dry with a maximum air pressure in order to obtain the thin film.
- In case of using other light-curing agents, use only those which have thin film thickness to ensure the good fit of the veneer.
- Light cure the bonding agent for the designated time in the instructions for use.

Luting procedure

- Select the suitable shade. Apply G-enial Universal Flo directly to the prepared surfaces of the restoration and/or tooth surface. Exposure to intensive light should be avoided using appropriate instructions.
- Keep moderate pressure on the restoration moderate pressure and remove excesses using one of the following techniques:
 - Keep moderate pressure and remove excesses using a brush. Make sure that excess are thoroughly removed from area of difficult access (e.g. interproximal areas).
 - Use a cotton swab to remove excesses using a brush. Make sure that excess are thoroughly removed from area of difficult access (e.g. interproximal areas).
- Thoroughly light-cure each surface of the restoration for 30 seconds.
- Remove the overlay/inhibition layer at the margins of the restoration, cover the margins with a glycerin gel before final light curing.

Polishing the margins

- Polish the restoration margins using appropriate instruments for the polishing of composite resins.

SHADES

A1, A2, A3, A3.5, A4, B1, B2, B3, C1, A02, A03, B0V, CV, JE, A.E. A, B, C, shades are based on the color of the tooth.

STORAGE

Recommended for optimal performance, store in a cool and dark place (4 - 25°C / 39.2 - 77°F) away from light.

PACKAGES

- Syringe (20mL), 20 dispensing tips (plastic type), 1 light protective cover
- 2, 30 dispensing tips (plastic type), 2 light protective covers
- 30 dispensing tips (needle type), 2 light protective covers

CAUTION

- In case of contact with tooth or skin, remove immediately with cotton or a sponge soaked in alcohol. Flush with water to remove any slurry. In a rubber dam and/or cocoa butter to be in contact to isolate the operation field from the skin or oral tissue.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Take care not to mix with other materials.
- Do not mix with other similar products.
- Dispensing tips are for single use only. To prevent cross contamination, use a new dispensing tip to avoid direct contact with an inhibited water layer in order to prevent possible sensitivity.
- Wear protective eye glasses during light curing.
- When using the overlay/inhibition layer, use a dust collector and wear a dust mask to avoid inhalation of cutting dust.
- Avoid getting material on clothes.

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