Prior to use, carefully read the instructions for use. **EN**

G-aenial™ Universal Injectable

UNIVERSAL LIGHT-CURED RADIOPAQUE INJECTABLE COMPOSITE

For use only by a dental professional in the recommended indications.

INDICATIONS FOR USE

- Direct restorative for Class I, II, III, IV and V cavities
- Fissure sealant
- Sealing hypersensitive areas
- Repair of (indirect aesthetic restorations, temporary crown & bridge, defect margins when margins are in enamel)
- Blockout of margins
- Linor or base
- Realisation of crowns & bridges, inlays and veneers using the indirect technique in combination with GRADIA or GRADIA PLUS component (please refer to their dedicated IFU).
- Splicing of teeth in combination with fibres such as GC everStick fibres. Follow manufacturer's instructions.

CONTRAINDICATIONS

- Direct pulp capping.
- Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.

DIRECTIONS FOR USE

- Shade Selection**
Select shade from 16 shades of XBW, BW, A1, A2, A3, A3.5, A4, B1, B2, CV, CVD, AO1, AO2, AO3, JE, AE.
A, B, C, D shades are available on Vita® Shade.
*Vita® is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany.
- Cavity Preparation**
Prepare cavity using standard techniques. Dry by gently blowing with oil free air. Rubber dam is recommended to isolate the prepared tooth from contamination with saliva, blood or sulcus fluid.
Note:
For pulp capping, use calcium hydroxide.

- Bonding**
For bonding G-aenial Universal Injectable to enamel and/or dentin, use a light-cured bonding system such as G-Premio BOND, G-aenial Bond or G-BOND. Follow manufacturer's instructions.
- Placement of G-aenial Universal Injectable**
1) Dispensing from the syringe
a. Hold the syringe upright and remove the wing cap by turning the counterlockwise (Fig. 1).
2) Tightly and securely attach the dispensing tip to the syringe by turning clockwise. Light protective cap. Place the light protective cap until use.

- Remove cap from the dispensing tip.**
Note:
Take care not to attach the dispensing tip too tightly. This may damage its screw. In order to ensure a light confection, make sure that the threads are free of remnant material. Prior to syringe material into the cavity, make sure to check the following:
a. Dispensing tip is securely attached to the syringe.
b. Any possible air from the dispensing tip is removed by gently pushing forward the conical plunger with the tip pointing upwards until material reaches the opening of the tip (Fig. 2).

- Place in cavity as close as possible**
a. Plug the cavity as close as possible to the prepared cavity, and slowly push the plunger to extrude material. Alternately, dispense material onto a mixing pad and transfer to the cavity using a suitable instrument.
Note:
a. If plugged, remove the dispensing tip and extrude a small amount of material directly from the syringe.
b. Minimize exposure to ambient light. Ambient light can shorten the manipulation time.
c. After use, immediately remove and dispose of the dispensing tip into a light-proof container with the wing cap.

- Dispensing from a Unitip**
Insert the G-aenial Universal Injectable Unitip into the Unitip APPLIER I or equivalent. Remove the cap and extrude material directly into the prepared cavity. Use steady pressure (Fig. 3). Maintain pressure on the applicator handle while removing the Unitip APPLIER I and Unitip from the mouth. This prevents the Unitip from coming loose from the applicator.
Note:
a. The material may be difficult to extrude immediately after removing from cold storage. Prior to use, leave to stand for a few minutes at room temperature.
b. For first use, apply pressure slowly and gradually to control extrusion of material.

- Clinical Hints**
In order to inject effectively, use the surface tension of the material to ensure uniformity across the entire surface of the restoration during build up. Once the required amount has been injected, release the pressure on the plunger and withdraw the syringe in a direction perpendicular to the surface. This will allow the material to separate from the dispensing tip and provides a smooth surface over the restoration.

- Light-cure**
Light-cure G-aenial Universal Injectable using a light curing unit. Keep light guide as close as possible to the surface. Refer to the following chart for Irradiation Time and Effective Depth of Cure.

	10 sec. (High Power LED $\geq 1200 \text{ mW/cm}^2$)	
	20 sec. (High Power LED $\ge 1200 \text{ mW/cm}^2$)	20 sec. (High Power LED $\ge 1200 \text{ mW/cm}^2$)
A1, A2, A3, B1, B2, JE, AE	2.5 mm	2.5 mm
XBW, BW, A3.5, A4, AO1, AO2, CV, CVD, AO3	2.0 mm	2.0 mm
AO3	1.5 mm	1.5 mm

- The material should be placed and light-cured in layers. For maximum layer thickness, please consult above table.
- Lower light intensity may result in inefficient curing or discoloration of the material.
- For the indirect technique, light-cure each layer for 30 seconds with LABOLIGHT LV-III, for 10 to 15 seconds with STEPLIGHT SL-I. Final cure for 3 minutes using LABOLIGHT LV-III.

- Finishing and Polishing**
Finish and polish using standard techniques.

- STORAGE**
XBW, BW, A1, A2, A3, A3.5, A4, B1, B2, CV, CVD, AO1, AO2, AO3, JE, AE
Recommended for optimal performance, store in a cool and dark place (4-25°C/39-277°F) away from high temperatures or direct sunlight.

- PACKAGING**
I. Syringe
1. Syringe 1.7 g (1.0 mL) x 1, Dispensing Tip Long Needle x 10, 1 Light-protective cap x 1
2. Syringe 1.7 g (1.0 mL) x 1, Dispensing Tip Needle x 5, Dispensing Tip Long Needle x 5, Light-protective cap x 1
3. Syringe 1.7 g (1.0 mL) x 2, Dispensing Tip Needle x 10, Dispensing Tip Long Needle x 10, Light-protective cap x 2
4. Dispensing Tip Refill
Dispensing Tip Long Needle x 30, Light-protective cap x 3
5. Dispensing Tip Refill
Dispensing Tip Long Needle x 30, Light-protective cap x 3
II. Unitips
1. Unitip 0.27 g (0.16 mL) x 15

- Note:**
All packaging types may not be available in all countries.

- CAUTION**
1. In case of contact with oral tissue or skin, remove immediately with cotton or a sponge soaked in alcohol. Flush with water. To avoid contact, a rubber dam and/or COCOA BUTTER can be used to isolate the operation field from the patient's mouth.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. Take care to avoid ingestion of the material.
4. Prior to starting the filling process, partially extrude paste outside of patient's mouth and away from the patient to ensure paste is flowing properly.
5. Do not mix with other similar products.
6. Dispensing tips are for single use only. To prevent cross contamination between patients, do not reuse the dispensing tip. The dispensing tip cannot be sterilized in an autoclave or chemically.
7. For eye contact reasons, Unitips are for single use only.
8. Wear plastic or rubber gloves during operation to avoid direct contact with air inhibited resin layers in order to prevent possible sensitivity.
9. Wear protective eye glasses during light curing.
10. When polishing the polymerized material, use a dust collector and wear a dust mask to avoid inhalation of cutting dust.
11. Avoid getting material on clothing.
12. In case of contact with unintended areas of tooth or prosthetic appliances, remove with instrument, sponge or cotton pellet before light-curing.
13. Do not use G-aenial Universal Injectable in combination with eugenol containing materials as eugenol may hinder G-aenial Universal Injectable from setting.
14. Do not wipe the dispensing tip with excessive alcohol as this may cause contamination of the paste.
15. Dispose of all waste in accordance with local regulations.
16. In rare cases the product may cause sensitivity in some people. If any such adverse cases are experienced, discontinue the use of the product and refer to a physician.
17. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should be worn during use.
18. Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarise yourself with the safety data sheets available at: <http://www.georgroup.com>. They can also be obtained from your supplier.

- CLEANING AND DESINFECTING**
MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged. DO NOT IMMERSSE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

- Warnings and Reporting:**
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please notify us immediately through the relevant vigilance system by selecting the proper authority of your country accessible through the following link:
https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental. In this way you will contribute to improve the safety of this product.

Last revised: 10/2020

2797

MANUFACTURED BY
GC DENTAL PRODUCTS CORP.
9300 Tomonaga-cho, Kasuga, Aichi 486-0844, Japan

GC CORPORATION
7-1-1 Haboru-cho, Nishiku, Tokyo 143-8585, Japan
1-1-1 Hongo, Bunkyo-ku, Tokyo 113-8505, Japan

GC EUROPE NV
Researchpark Hearsele-Lieven 1240, Herestraatweg 33, 3001 Leuven, Belgium

GC ASIA DENTAL PTE. LTD.
5 Tampines Central 1, #06-01 Tampines Plaza, Singapore 529541

GC AUSTRALASIA DENTAL PTY. LTD.
1753 Botany Rd, Bankmeadow, NSW 2019, Australia
TEL: +61 2 9301 8300

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