Prvor se koristite, carefully read the instructions for use.

**EN**

**Instruction for Use for REFILLS**

**GRADIA™ PLUS MODULAR COMPOSITE SYSTEM FOR INDIRECT RESTORATIONS**

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**INDICATIONS**

- Crown and Bridgework (with metal backing).
- Metal free restorations: Jacket Crowns, Inlays, Onlays and Laminated Veneers.
- Implant superstructures.
- Replacement of gum tissue for crown restorations.

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IRADIATION TIME AND CURING UNIT				
Curing unit	LABOLIGHT DUO	STEP-LIGHT SL-I	LABO-LIGHT LV-III	
	Step- mode**	Full- mode	Pre-Cure**	Final cure
OPACITE				
PASTE HB, PASTE LB, GUM SHADES LB, GUM SHADES HB	10 sec.*	3 min.	10 sec.*	3 min.
LUSTRE PAINT***	10 sec.*	90 sec.	10 sec.*	5 min.

\* For one surface of a single crown.
\*\* Distance from light source : 3cm.
\*\*\* Thickness: 0.1 mm or less.

**CAUTION**

- In rare cases, this product may cause sensitivity in some people. If such reactions are experienced, discontinue the use of the product and consult a physician.
- Wear eye protection and plastic or rubber gloves during operation in particular to avoid direct contact with paste and air inhibited resin layers in order to prevent possible sensitivity.
- When using this product, do not use heat when adjusting or polishing the veneer surface, use a dust collector and wear a dust mask to avoid inhaling dust.
- In case of contact with oral tissues or skin, immediately remove with cotton. Flush with water.
- In case of contact with eyes, immediately wash the eye with copious amounts of water and seek medical attention.
- This product should not be used for malocclusion, clenching or bruxism.
- At the time of cementation, sandblast the inner surface of metal free restorations, and then be sure to treat the inner surface with a silane coupling agent (e.g. CERAMIC PRIMER II (GC)).
- Dispose all waste according to local regulation.
- To prevent cross contamination between patients, do not reuse the dispensing tip.
- The dispensing tip cannot be sterilized in an autoclave or chemically.
- Wear protective eyeglasses during light curing. An ultra-light intensity may cause insufficient curing.

**STORAGE**

Recommended for optimal performance, store in a cool place (4-25°C / 39-27.0 °F) away from high temperatures or direct sunlight.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com>

They can also be obtained from your supplier.

Undesired effects- Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report by directy through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: [https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en) as well as to our internal vigilance system: [vigilance@gc.dental](mailto:vigilance@gc.dental)
In this way you will contribute to improve the safety of this product.

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Viimati redigeeritud: 07/2019

Prior to use, carefully read the instructions for use.

EN

## Instruction for Use for REFILLS

### GRADIA® PLUS MODULAR COMPOSITE SYSTEM FOR INDIRECT RESTORATIONS

For use only by a dental professional in the recommended indications.

#### INDICATIONS

1. Crown and Bridgework (with metal backing).
2. Metal free restorations: Jacket Crowns, Inlays, Onlays and Laminated Veneers.
3. Implant superstructures.
4. Reproduction of gum tissue for crown restorations.

#### CONTRAINDICATIONS

Avoid use of this product in patients with known methacrylate monomer and methacrylate polymer allergies.

#### IRRADIATION TIME AND CURING UNIT

Curing unit	LABOLIGHT DUO		STEP- LIGHT SL-I	LABO-LIGHT LV-III
	Step-mode**	Full-mode	Pre-Cure**	Final cure
OPAQUE	-	1 min.	-	1 min.
PASTE HB, PASTE LB, GUM SHADES LB, GUM SHADES HB	10 sec.*	3 min.	10 sec.*	3 min.
LUSTRE PAINT***	10 sec.*	90 sec.	10 sec.*	5 min.

\* For one surface of a single crown.

\*\* Distance from light source : 3cm.

\*\*\* Thickness: 0.1mm or less.

#### CAUTION

1. In rare cases, this product may cause sensitivity in some people. If such reactions are experienced, discontinue the use of the product and consult a physician.
2. Wear eye protection and plastic or rubber gloves during operation in particular to avoid direct contact with paste and air inhibited resin layers in order to prevent possible sensitivity.
3. LUSTRE PAINT DILUTING LIQUID, SEPARATOR, DIE-HARDNER, GC METAL PRIMER II / GC METAL PRIMER Z and GC CERAMIC PRIMER II are highly flammable liquid and vapour. Do not use near naked flame. Keep away from sources of ignition. Keep away from direct sunlight.
4. After dispensing, use immediately and replace caps on syringe or bottle.
5. When adjusting or polishing the veneer surface, use a dust collector and wear a dust mask to avoid inhaling dust.
6. In case of contact with oral tissues or skin, immediately remove with cotton. Flush with water.
7. In case of contact with eyes, immediately wash the eye with copious amounts of water and seek medical attention.
8. This product should not be used for malocclusion, clenching or bruxism.
9. At the time of cementation, sandblast the inner surface of the metal free restorations, and then be sure to treat the inner surface with a silane coupling agent (e.g. CERAMIC PRIMER II (GC)).
10. Dispose all waste according to local regulation.
11. To prevent cross contamination between patients, do not reuse the dispensing tip.
12. The dispensing tip cannot be sterilized in an autoclave or chemiclave.
13. Wear protective eye glasses during light curing.
14. Lower light intensity may cause insufficient curing.

#### STORAGE

Recommended for optimal performance, store in a cool place (4-25°C / 39.2-77.0 °F) away from high temperatures or direct sunlight.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com> They can also be obtained from your supplier.

Undesired effects- Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: [https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en) as well as to our internal vigilance system: [vigilance@gc.dental](mailto:vigilance@gc.dental) In this way you will contribute to improve the safety of this product.

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MANUFACTURED by GC DENTAL PRODUCTS CORP. 2-285 Torimatsu-cho, Kasugai, Aichi 486-0844, Japan

RESPONSIBLE MANUFACTURER IN CANADA: GC AMERICA INC. 3737 West 127th Street, Alsip, IL 60803, U.S.A.

DISTRIBUTED by: GC CORPORATION 76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU : GC EUROPE N.V. Researchpark Haasrode-Luvenen 1.240, Interleuvenlaan 33, B-3001 Leuven, Belgium TEL:+32 16 74 10 00

GC AMERICA INC. 3737 West 127th Street, Alsip, IL 60803 U.S.A. TEL: +1-708-597-0900. [www.gcamerica.com](http://www.gcamerica.com)

GC ASIA DENTAL PTE. LTD. 5 Tampines Central 1, #06-01 Tampines Plaza 2, Singapore 529541 TEL: +65 6546 7588

GC AUSTRALIA DENTAL PTY. LTD. 1753 Botany Rd, Banksmeadow, NSW 2019, Australia TEL: +61 2 9301 8200

PRINTED IN BELGIUM

952067\_NZ6302  
30000945  
IFU Gradia Plus WEP (Short) Liquids

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