GUTTAFUSION[®]

System

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EN Operating Manual

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ES Manual de instrucciones

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DA Brugsanvisning

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Table of contents

| | | |
|--------|--|----|
| I. | General information | 16 |
| II. | Step-by-step instructions | 16 |
| 1. | Crosslinking | 16 |
| 2. | Hydraulics | 17 |
| 3. | Shaping | 17 |
| 4. | Selection process of the correct GUTTAFUSION® Obturator or GUTTAFUSION® for RECIPROC® Obturator | 17 |
| 4.1 | When using NiTi Systems with a sequence of several files, such as Mtwo® or a hybrid technique | 17 |
| 4.1.1. | Verify the shape at working length and passive fit | 17 |
| 4.1.2. | Irrigate the canal | 18 |
| 4.1.3. | Selecting the right obturator | 18 |
| 4.2. | When using RECIPROC® instruments: GUTTAFUSION® for RECIPROC® | 18 |
| 4.2.1. | Irrigate the canal | 19 |
| 4.2.2. | Selecting the right obturator | 19 |
| 4.3. | Table of corresponding preparation sizes and GUTTAFUSION® Obturators | 19 |
| 5. | Asepsis | 20 |
| 6. | Drying the canal and applying sealer | 20 |
| 7. | Heating GUTTAFUSION® Obturators in the GUTTAFUSION® Oven | 20 |
| 8. | Placing obturator in the canal | 21 |
| 9. | Removing the GUTTAFUSION® Obturator shaft and handle | 21 |
| 10. | Compacting gutta-percha in large, oval or irregular shaped canals | 21 |
| 11. | Removing excess gutta-percha | 22 |
| 12. | Removing GUTTAFUSION® obturation materials | 22 |
| III. | Symbols used on packaging, device and parts | 23 |

I. General information

FOR DENTAL USE ONLY!

Description

GUTTAFUSION® Obturators are used for filling root canal systems.

Indication for use

GUTTAFUSION® Obturators are used to fill the shaped, cleaned and irrigated root canal space.

Contraindications

None known.

Warnings and adverse reactions

This product may contain dry natural rubber. Patients with latex sensitivity may experience allergic reactions to gutta-percha, which may contain dry natural rubber.

Precautions

1. Prior to using any new product, read all of the manufacturer's instructions. Make sure that you are competent when using the product and technique prior to clinical use.
2. Please make sure that you have prepared and cleaned your root canal of debris and carefully irrigated before working with GUTTAFUSION® Obturators. A good straight line access to the canals is recommended.
3. Select the GUTTAFUSION® Obturator according to the GUTTAFUSION® Size Verifier that fits the shaped canal passively (see 4.1. for details).
4. When shaping the canal with RECIPROC® instruments only, use GUTTAFUSION® for RECIPROC® (see 4.2. for details).
5. GUTTAFUSION® Obturator cores cannot be used as size verifiers.
6. After heating the GUTTAFUSION® Obturator in the GUTTAFUSION® Oven avoid contact with skin, mucosa etc., because the obturator will be hot.

7. Do not remove gutta-percha from the GUTTAFUSION® Obturator prior to filling the root canal space. Removing the gutta-percha damages the obturator.

Disposal consideration

Observe all federal, state and local environmental regulations.

II. Step-by-step instructions

These instructions are designed to help gain an understanding of the technique and safe application of the product. Practice on plastic blocks and extracted teeth is strongly recommended prior to clinical use.

Prior to the application of GUTTAFUSION® prepare, clean and dry the root canal.

1. Crosslinking

Crosslinking is a well-known chemical process of linking polymer chains together. Crosslinking creates the subtle strength needed in the core of a centrally compacting obturator. GUTTAFUSION® Obturators create the hydraulics needed with the added benefit of having a gutta-percha core. It is important to note that crosslinked gutta-percha is not sticky, does not melt and will not dissolve with solvents.

2. Hydraulics

GUTTAFUSION® Obturators move warm gutta-percha 3-dimensionally into the root canal system. Many obturation techniques use lateral or vertical compaction, however the hydraulic force from these techniques generally moves gutta-percha in one or two unequal directions (laterally or apically). GUTTAFUSION® uses a crosslinked gutta-percha core for centrally compacting the flowing gutta-percha in the canal. The hydraulic force compacts warm gutta-percha flowing equally in 3-dimensions. Proper cleaning, shaping and irrigation, along with the central compacting hydraulic force facilitates the flow of gutta-percha in the root canal system.

3. Shaping

Proper shaping removes pulpal tissue and dental debris permitting the GUTTAFUSION® Obturator to move passively through the canal.

Required shape:

- i) Straight-line access for straight insertion without striping gutta-percha off the carrier.
- ii) Ensure the GUTTAFUSION® Obturator has sufficient space to enter the root canal along the straight line access and if necessary flare the canal coronally.
- iii) Prepare greater tapered shape in the apical part (at least .04) and preserve the apical constriction.

4. Selection process of the correct GUTTAFUSION® Obturator or GUTTAFUSION® for RECIPROC® Obturator

Proper shaping, cleaning and drying of the root canal is an important key to success when using GUTTAFUSION® Obturators.

4.1 When using NiTi Systems with a sequence of several files, such as Mtwo® or a hybrid technique

4.1.1. Verify the shape at working length and passive fit

Following proper shaping and cleaning, confirm the final working length measurement using a GUTTAFUSION® Size Verifier made of nickel-titanium, that reaches the apical constriction passively, with no significant resistance or twisting. It is necessary to check passive fit of the GUTTAFUSION® Size Verifier at working length in order to confirm the shape of the canal and the correct choice of GUTTAFUSION® Obturator. Verifying the shape at working length and passive fit ensures that the GUTTAFUSION® Obturator being used matches the correct apical canal diameter.

NOTE

If the root canal was prepared with a taper.05 or higher, in most cases the GUTTAFUSION® Size Verifier of the same ISO size as the last file taken to working length will fit passively. If the last file taken to working length has a taper.04 it is more likely that a GUTTAFUSION® Size Verifier one size smaller will fit passively.

Verifying the passive fit: Take a GUTTAFUSION® Size Verifier of the same ISO size as the last file taken to working length and confirm a passive fit at working length; set the silicone stopper at your working length using the size verifier's calibration mark. Check that the size verifier goes passively into working length (no twisting; no significant resistance) and confirm by rotating in the canal 180°:

- a. If the fit is passive, please irrigate your canal (see 4.1.2 "Irrigate the canal") before proceeding to 4.1.3 "Selecting the right obturator".
- b. If the fit is not passive, select a smaller size verifier which fits the apex passively. If the smaller size verifier fits passively, irrigate your canal carefully (see 4.1.2 "Irrigate the canal").

4.1.2. Irrigate the canal

The cleaning of the shaped root canal system is accomplished primarily through the use of copious irrigation with NaOCl. Irrigants can be activated using a cavitation and acoustic streaming device or ultrasonic irrigation needle. The activated irrigants improve debridement and disrupt the smear layer and biofilm inside the canal, promoting deep cleaning and disinfection of lateral canals, fins, webs, isthmuses, anastomoses, and dentinal tubules. Dry the canal thoroughly using absorbent paper points.

4.1.3. Selecting the right obturator

Make sure that the fit of the size verifier used is passive. If the fit of the size verifier used is passive, take a GUTTAFUSION® Obturator of the same size. (See also 4.3 "Table of corresponding preparation sizes and GUTTAFUSION® Obturators".)

Set the silicone stopper on the GUTTAFUSION® Obturator to the working length established. Place the bottom of the stopper at the top of the calibration ring. Calibration rings are set at the following working lengths (in millimeters): 18, 19, 20, 22, 24.

Please proceed to step 5.

4.2. When using RECIPROC® instruments: GUTTAFUSION® for RECIPROC® GUTTAFUSION® for RECIPROC® blue

NOTE

If you combine the RECIPROC® system with another file or files (i.e. a hybrid technique), please follow the instructions described in step 4.1 of this DFU.

Contrary to NiTi Systems with a sequence of several files, with RECIPROC® just one instrument is used to prepare the root canal to full working length, so the RECIPROC® instrument itself acts as a Verifier. After preparation and thorough irrigation of the root canal, select the GUTTAFUSION® for RECIPROC® obturator according to the RECIPROC® instrument used (see 4.3. Table of corresponding preparation sizes and GUTTAFUSION® Obturators).

NOTE

During the standard preparation of the root canal with RECIPROC® instruments, brushing motions can enlarge the middle and coronal part of the root canal if necessary, particularly in case of oval or dumbbell shaped anatomies. This does not affect the selection of the obturator for the root canal. (For more information see step 10.)

4.2.1. Irrigate the canal

The cleaning of the shaped root canal system is accomplished primarily through the use of copious irrigation with NaOCl. Irrigants can be activated using a cavitation and acoustic streaming device or ultrasonic irrigation needle. The activated irrigants improve debridement and disrupt the smear layer and biofilm inside the canal, promoting deep cleaning and disinfection of lateral canals, fins, webs, isthmuses, anastomoses, and dentinal tubules. Dry the canal thoroughly using absorbent paper points.

4.2.2. Selecting the right obturator

For root canals shaped with RECIPROC® instruments only where the apical size gauged is an ISO 25, 40 or 50, special GUTTA FUSION® for RECIPROC® Obturators are available for filling these root canals.

Set the silicone stopper on the GUTTA FUSION® Obturator to the working length established. Place the bottom of the stopper at the top of the calibration ring. Calibration rings are set at the following working lengths (in millimeters): 18, 19, 20, 22, 24.

4.3. Table of corresponding preparation sizes and GUTTA FUSION® Obturators

NOTE

For root canals prepared with a sequence of several files, always use GUTTA FUSION® Size Verifiers to check obturator size required, see step 4.1.

RECIPROC®

| Final RECIPROC® blue Instrument | GUTTA FUSION® for RECIPROC® blue Obturator |
|---------------------------------|--|
| RECIPROC® blue R25 | R25 |
| RECIPROC® blue R40 | R40 |
| RECIPROC® blue R50 | R50 |

| Final RECIPROC® Instrument | GUTTA FUSION® for RECIPROC® Obturator |
|----------------------------|---------------------------------------|
| RECIPROC® R25 | R25 |
| RECIPROC® R40 | R40 |
| RECIPROC® R50 | R50 |

Mtwo®

| Final Mtwo® Instrument | GUTTA FUSION® Size Verifier |
|------------------------|-----------------------------|
| Mtwo® 25/06 | 25 |
| Mtwo® 25/07 | 25 |
| Mtwo® 30/06 | 30 |
| Mtwo® 35/06 | 35 |
| Mtwo® 40/06 | 40 |
| Mtwo® 30/05 | 30 |
| Mtwo® 35/04 | 30 |
| Mtwo® 40/04 | 35 |
| Mtwo® 45/04 | 40 |
| Mtwo® 50/04 | 45 |
| Mtwo® 60/04 | 55 |

Rotary systems

| Final Instrument .04 Taper | GUTTAFUSION® Size Verifier |
|-------------------------------|-------------------------------|
| 20/.04 | - |
| 25/.04 | 20 |
| 30/.04 | 25 |
| 35/.04 | 30 |
| 40/.04 | 35 |
| 45/.04 | 40 |
| 50/.04 | 45 |
| 55/.04 | 50 |
| 60/.04 | 55 |

| Final Instrument .06 Taper | GUTTAFUSION® Size Verifier |
|-------------------------------|-------------------------------|
| 20/.06 | 20 |
| 25/.06 | 25 |
| 30/.06 | 30 |
| 35/.06 | 35 |
| 40/.06 | 40 |
| 45/.06 | 45 |
| 50/.06 | 50 |
| 55/.06 | 55 |

5. Asepsis

Before heating, disinfect the obturator in a sodium hypochlorite solution for one minute. Gently wipe the gutta-percha on the GUTTAFUSION® Obturator with 70% alcohol.

6. Drying the canal and applying sealer

Use a sterile paper point corresponding to the size of the (final) file taken to working length. Brush a very light coating of 2Seal®easymiX® sealer in the root canal with a new sterile paper point. To help ensure the obturator reaches working length, use an additional absorbent point to remove any excess sealer that may have accumulated on the canal walls or pooled in the apex.

If obturating more than one canal in a single tooth, place sealer in all the canals at the same time. This facilitates removal of excess gutta-percha, should this be necessary, by preventing the gutta-percha from sticking to other orifices or dentin.

7. Heating GUTTAFUSION® Obturators in the GUTTAFUSION® Oven

Details on heating the GUTTAFUSION® Obturators or the GUTTAFUSION® for RECIPROC® Obturator are given in the GUTTAFUSION® Oven Directions for Use.

If you use a GuttaMaster® Oven, please heat all GUTTAFUSION® Obturators on the lowest setting (25 and lower). Do not use the other settings.

8. Placing obturator in the canal

After the first signal “beep” of the GUTTAFUSION® Oven, the obturator is ready for use. Push the obturator holder and guide the holder with your finger. Take the obturator carefully out of the holder by first lifting it up a little and then pulling it toward you. The obturator handle can be touched either with fingers or with standard tweezers.

The use of tweezers better facilitates the application in molars. Insert the GUTTAFUSION® Obturator directly into the orifice of the canal and guide the obturator to working length with a slow, steady, apically directed movement. To prevent the carrier from bending, it is important that the movement is always directed along the longitudinal axis of the carrier. On insertion, the gutta-percha flows apically in front of the carrier. This allows filling up to the apex, as well as sealing of side canals and isthmi.

After having reached the working length, the pressure should be maintained for a few seconds, for the gutta-percha to cool off under light pressure.

NOTE

If the canal has been properly shaped and the gutta-percha properly heated, the obturator should seat to place without twisting or forcing. To avoid overextension, do not force the GUTTAFUSION® Obturator beyond the working length. You will note a backflow of sealer and gutta-percha resulting in an accumulation at the orifice. This is to be expected, especially in multi-rooted teeth, because the obturator is designed with excess gutta-percha to accommodate even the most widely flared or anatomically irregular canals.

9. Removing the GUTTAFUSION® Obturator shaft and handle

Remove the shaft and handle at the orifice by bending to either side of the canal wall. Alternatively, while stabilizing the GUTTAFUSION® Obturator with your index finger, use a round bur, or an inverted cone bur in a high-speed hand-piece, or use a sharp spoon excavator.

NOTE

The gutta-percha carrier cannot be melted off because the material is resistant to heat.

10. Compacting gutta-percha in large, oval or irregular shaped canals

Insert a small segment of conventional gutta-percha, 4-6 mm long, alongside the shaft of the GUTTAFUSION® Obturator with a lubricated plugger and compact. The gutta-percha segment compacts the heat softened gutta-percha and becomes an integral part of the filling. Due to the tacky nature of the plasticised gutta-percha, a lubricant such as 2Seal®easymiX® root canal sealer, topical anesthetic, etc., can be used on the plugger. This will prevent inadvertent removal of gutta-percha from the orifice with an inadequately lubricated plugger, as well as compensate for the lack of gutta-percha in uncommonly large canals or those with internal resorption defects.

11. Removing excess gutta-percha

Using a spoon excavator, explorer, etc., remove any excess gutta-percha that may block access to the chamber and other canals. Repeat all of the above steps on each canal of a multi-rooted tooth. If necessary, protect adjacent canal orifices from debris and gutta-percha with tiny cotton pellets or a paper point until time for obturation. Obturation is complete once excess gutta-percha is removed.

12. Removing GUTTAFUSION® obturation materials

en Creating post space and retreatment are accomplished using traditional methods for removing obturation material. For creating post space, remove the GUTTAFUSION® Obturator by selecting either an appropriately sized post drill of the root post to be used, e.g. DT Universal drill, Gates Glidden or Peeso Reamer. When removing the obturation material for retreatment purposes, use either an Mtwo® retreatment instrument R25/.05 or RECIPROC® instrument R25. Take a radiograph to confirm the removal of all obturation material.

If retreating and encountering intricate anatomy in the apical 1/3 of the canal, use a hand file with solvent to soften the gutta-percha on the GUTTAFUSION® Obturator.

III. Symbols used on packaging, device and parts

Symbols used for GUTTAFUSION® Obturators

Symbols used for GUTTAFUSION® Size Verifiers

| | | | |
|---|--------------------------------------|---|--------------------------------------|
|  | Consult instructions for use |  | Consult instructions for use |
|  | Do not re-use |  | Do not re-use |
|  | LOT Number |  | LOT Number |
|  | Manufacturer |  | Manufacturer |
|  | Length |  | Length |
|  | CE marking |  | CE marking |
|  | Use by date |  | Date of manufacture |
|  | Opened packages will not be replaced |  | Opened packages will not be replaced |
|  | Non sterile |  | Non sterile |
|  | Packaging unit |  | Packaging unit |
|  | Catalogue number (re-order number) |  | Catalogue number (re-order number) |
|  | Keep away from sunlight and heat |  | Autoclavable |

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**Manufacturer: VDW GmbH**