

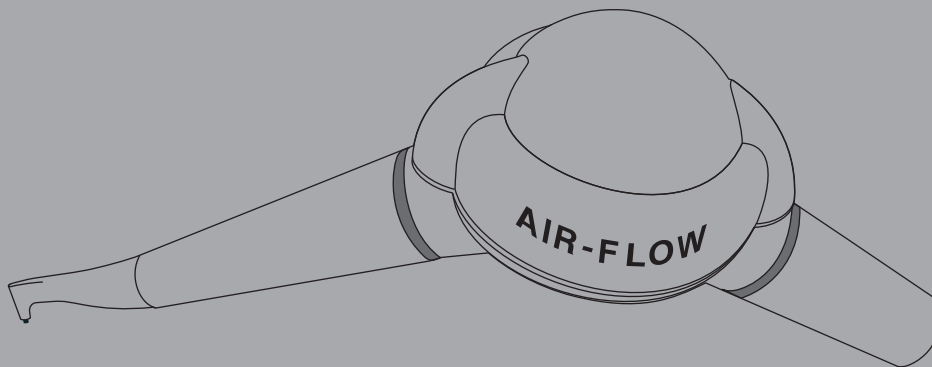
OPERATION INSTRUCTIONS  
MODE D'EMPLOI  
GEBRAUCHSANWEISUNG  
INSTRUCCIONES DE FUNCIONAMIENTO  
ISTRUZIONI PER L'USO  
BRUKSANVISNING

# AIR-FLOW<sup>®</sup> handy 2+

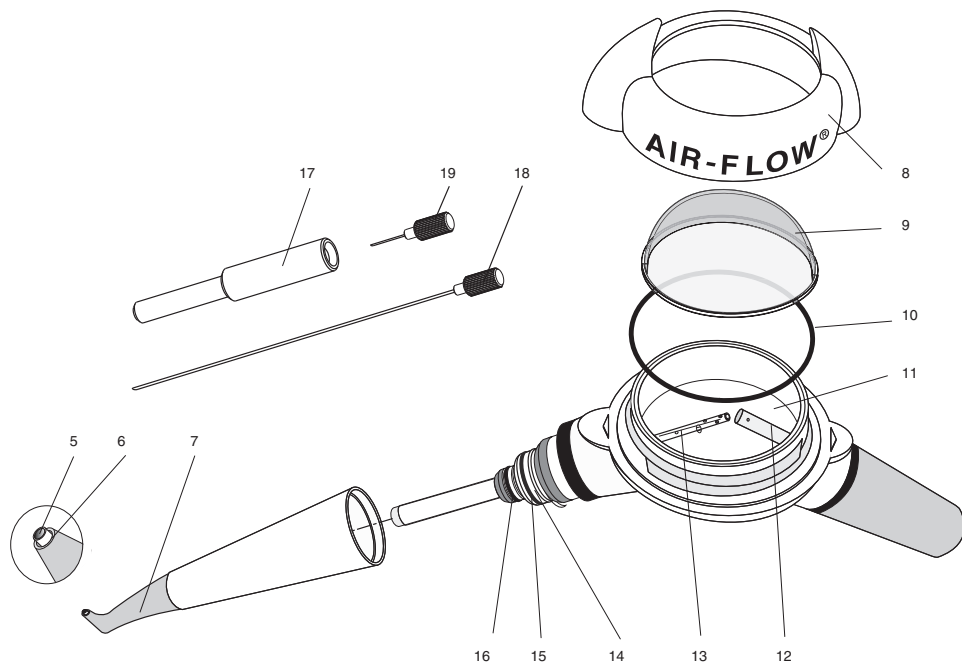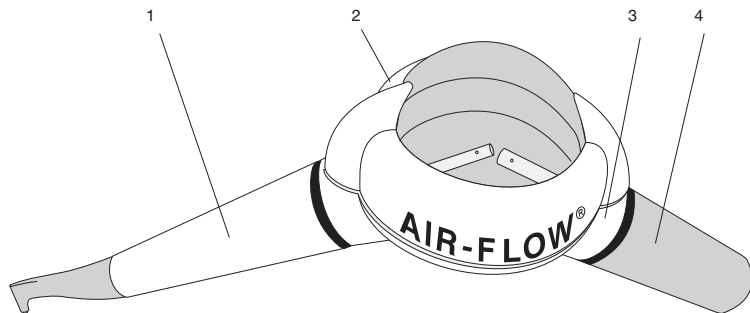**Rx ONLY** Caution! Federal (USA) law restricts this device to sale by or on the order of a dentist.

**EMS**   
ELECTRO MEDICAL SYSTEMS

## COMPONENTS

- 1 Handpiece
- 2 Powder chamber cap
- 3 Body
- 4 Connector
- 5 Powder outlet tube
- 6 Water outlet port
- 7 Nozzle
- 8 Cap ring
- 9 Cap dome
- 10 Powder chamber seal
- 11 Powder chamber
- 12 Rear tube
- 13 Front tube
- 14 Handpiece connection
- 15 Large O-ring for handpiece connection
- 16 Small O-ring for handpiece connection
- 17 Needle guide
- 18 Long cleaning needle
- 19 Short cleaning needle

EMS supplies units with various accessories. The "Packing list" shows exactly what your unit includes.

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## DEAR CUSTOMERS,

Thank you for buying a new EMS product. It meets the highest quality and safety standards.

This system is for use in dentistry with EMS AIR-FLOW units.

This product functions with the powder provided by EMS.

The confinement of the powder by a water jet makes it possible to steer the water jet with great precision and thus ensures a pleasant treatment for your patient.

This product removes dental plaque, soft deposits and surface stains from pits, grooves, interproximal spaces or smooth surfaces of the teeth.

### Applications

- Plaque removal for placement of sealants,
- Surface preparation prior to bonding/cementation of inlays onlays, crowns and veneers,
- Surface preparation prior to placing composite restorations,
- Cleaning prior to bonding orthodontic brackets,
- Effective plaque and stain removal for orthodontic patients,
- Clean implant fixture prior to loading,
- Stain removal for shade determination,
- Plaque removal prior to fluoride treatment,
- Plaque and stain removal prior to whitening procedure.

### Contra-indication

Patients suffering from chronic bronchitis or asthma must not, under any circumstances, be treated with an air polishing device. The jet of air and powder could cause respiratory difficulties.

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## ABOUT THIS MANUAL

These operating instructions are to ensure the correct installation and use of this product.

 Please read these operating instructions carefully as they explain all the most important details and procedures. Please pay special attention to the safety precautions.

Always keep this document close to hand.

To prevent injury to people and damage to property, please heed the corresponding directives. They are marked as follows:

 **Danger**  
Risk of patient or user injury

 **Caution**  
Risk of damage to the device or environmental harm

 **Please note**  
Useful additional information and hints

 **Prohibited**

 **Authorized**

## Technical modifications

EMS reserves the right to modify the technique, accessories, operating instructions or contents of the set due to technical or scientific improvements.

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EMS

## CONTENTS

General hints.....	2
Information about powders .....	2
Setting the dental unit .....	4
Installation and set up.....	6
Checking the turbine connection (typical is shown).....	6
Connecting to the dental unit .....	8
Setting the water flow rate .....	10
Filling the powder chamber .....	12
Closing the cap .....	14
Handling and setting water/air .....	16
Precautions .....	18
Wear a mask and eye protection .....	18
Treatment.....	22
Working Technique .....	24
Precautions to be taken at the end of the treatment .....	28
Fluoride Application .....	30
Patient Information .....	32
Cleaning, disinfecting and sterilizing .....	34
Unit .....	34
Handpiece .....	36
Daily cleaning of the device .....	38
Daily cleaning of the cap.....	40
Maintenance.....	42
Replacing the dome and the O-rings.....	42
Safety precautions .....	44
Storing the product .....	46
Product disposal .....	48
Warranty.....	48
Accessories.....	48
EMS-Service.....	50
Symbols .....	52
Technical data .....	53
Troubleshooting .....	56

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## GENERAL HINTS

### Information about powders

 Patients on a low salt diet must not be treated with the powder containing sodium bicarbonate. For patients on a low salt diet use the powder without sodium bicarbonate provided by EMS.

 The aromatic powder provided by EMS can lead, in specific cases, to allergic reactions to the flavor. For patients known to suffer such reactions, use the powder without flavor provided by EMS.

 Do not direct the jet of powder toward fillings, crowns or bridgework as this could damage these restorations.

 The powder containing sodium bicarbonate provided by EMS can be used only for supra-gingival application. For sub-gingival application, please use the specific powder provided by EMS and refer to its operation instructions.

 Never use an abrasive powder i.e. silica or alumina based. This would damage the unit.

 The powders provided by EMS are especially designed for use with the unit. Do not use powders from other manufacturers as this could damage the unit or could adversely affect its efficiency.

 For more details about the EMS powder, please read the relative operation instructions.

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## INSTALLATION AND SET UP

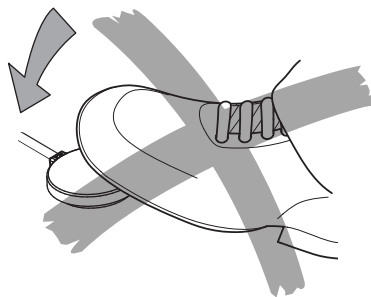
Checking the turbine connection  
(typical is shown)

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⚠ The device is equipped with a connector specially designed for the turbine connection of your dental unit. Only use the device with this special turbine connection. To connect it to another type of turbine will damage it.

⚠ The turbine of your dental unit must not be under pressure when connecting the device. Do not activate the turbine footpedal. If you have a turbine equipped with light, turn the light off.

⚠ Make sure the O-rings of your turbine connection are in good condition. A turbine connection with O-rings in bad condition can damage the device.

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



## Filling the powder chamber

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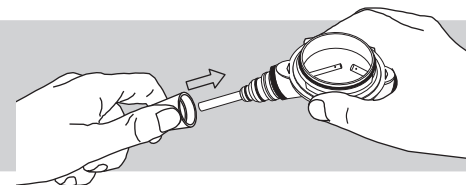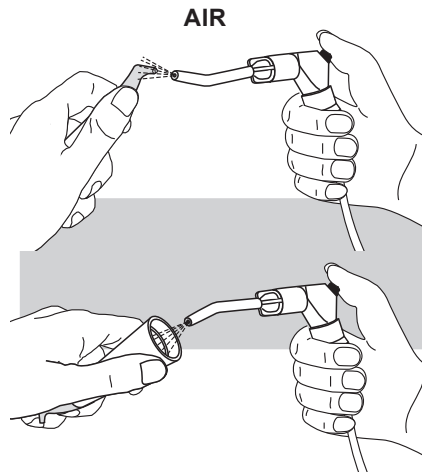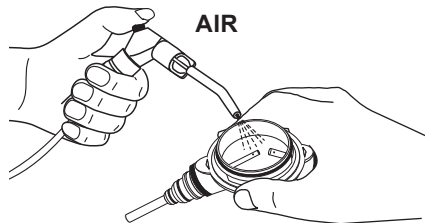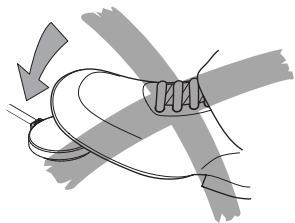⚠ Make sure that the powder chamber is absolutely dry. Moisture can cause the powder to cake.

⚠ After sterilization, moisture may be present in the handpiece. Blow the interior of the handpiece with the compressed air to avoid the formation of powder concretions in the air tubes.

⚠ Use only powder provided by EMS.

⚠ Do not go over the "max." limit. The openings of the tubes should not be covered with powder. The tubes could get clogged.

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 The contact lenses or glasses of the patient may get dirty during the treatment. We recommend to take them off.

 To limit the risk of bacterial or viral infections and inhalation of powder, we recommend to the dentist and the hygienist to wear a safety protection mask.

 A powder jet accidentally directed into the eye can cause severe ocular injury. Therefore it is strongly recommended for all persons to wear eye protection during treatment.

 The nozzle should never be pointed toward the patient before, during and after operation.

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

## Working Technique

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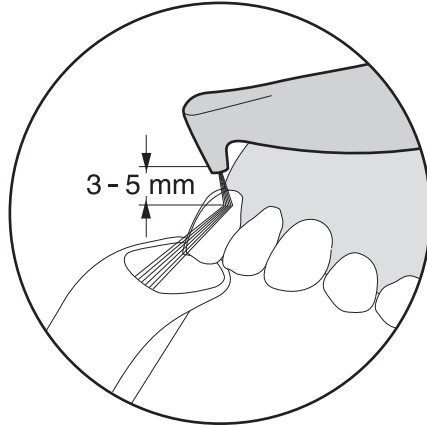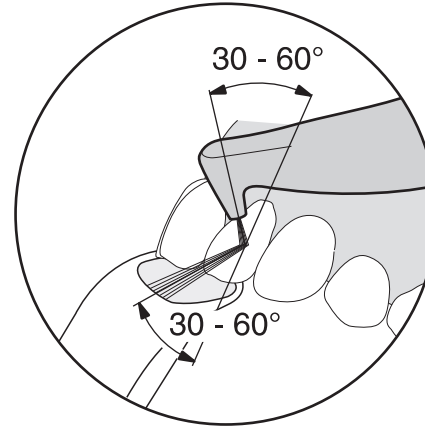 In subgingival applications, use the EMS powders for subgingival application and refer to the instructions for use delivered with the product.

 Direct the nozzle strictly towards the surface of the tooth. Respect a distance of 3 to 5 mm.

 You can vary the angle between nozzle and tooth from 30 to 60 degrees. The flatter is the angle, the larger is the area cleaned.

 During treatment, direct the high-speed pump towards the air/powder jet deviated by the tooth. The angle of reflection is identical to the angle of incidence.

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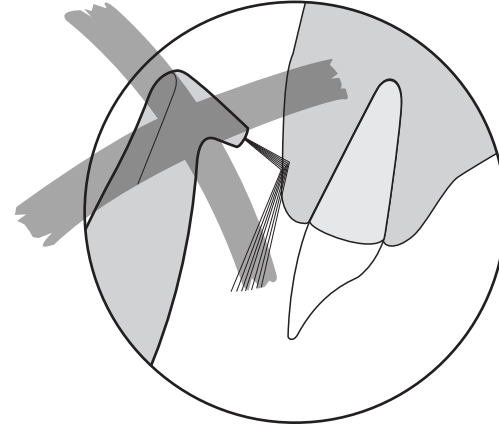
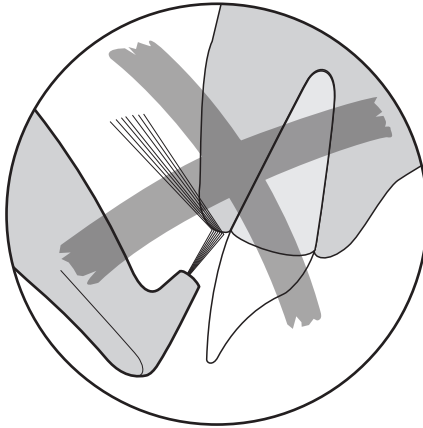
 The air/powder jet is powerful. It can cause injury to the gums or an emphysema caused by the introduction of air into the soft tissue spaces. It is strongly recommended that the operator never direct the nozzle directly at the gum tissue or into the gingival sulcus.

 During treatment, make small circular motions.

Finally after the treatment polish all dental surfaces by setting the water flow rate to the maximum.

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
## Precautions to be taken at the end of the treatment

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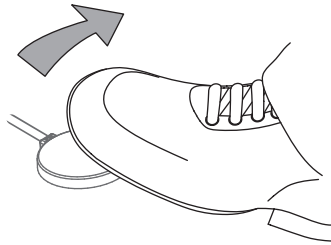SEC...

Release the pedal and wait until the spray stops.

 When removing the foot from the control pedal, the air/powder jet continues for a few more seconds. Take these seconds into account.

 You can also introduce the nozzle still in the patient mouth into the high-speed evacuator. Let some time for the unit to decompress, with no risk of damaging patient mouth.

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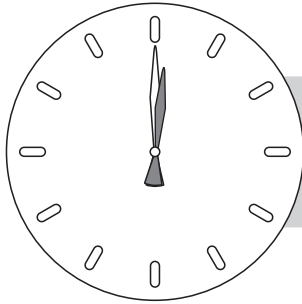
## Information au patient

## Patientenunterweisung

 After the treatment, the teeth are clean and the dental cuticle is completely eliminated. Its reconstitution by salivary proteins requires between 2 and 3 hours. During this time, the teeth do not have any more natural protection with respect to coloring.

 Tell your patient that during 2 to 3 hours following treatment, he should neither smoke, nor consume food or drinks which could strongly color the teeth (tea, coffee...).

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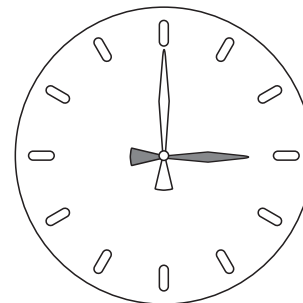
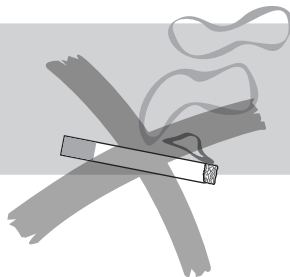
## CLEANING, DISINFECTING AND STERILIZING

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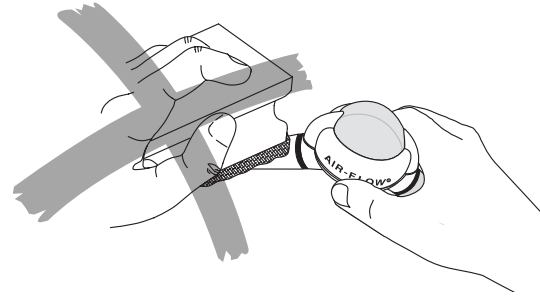
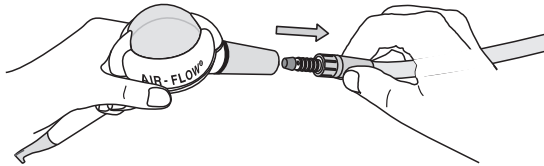
Gerät

⚠ Clean the unit only with an alcohol-based, commercially available (ethanol, isopropanol), colorless disinfectant. The use of scouring powder or an abrasive sponge will damage its surface.

⚠ Do not submerge the unit in a disinfectant bath. It would be likely to be damaged.

⚠ The unit is not protected against water spray. It cannot be sterilized.

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## SAFETY PRECAUTIONS

EMS and the dealer of this product accept no liability for direct or consequential injury or damage resulting from improper use, arising in particular through non-observance of the operating instructions, or improper preparation and maintenance.

 Before each use clean, disinfect and sterilize the different pieces and accessories of this product. Please refer to the information provided in the operating instructions. Non-sterile pieces and accessories may cause bacterial or viral infections.

 Use for the intended purpose only: Before using the product, make sure that you have studied and understood the operating instructions. This also applies to any equipment used with this product. Failure to observe the operating instructions may result in the patient or user suffering serious injury or the product being damaged, possibly beyond repair.

 This product must be used only by trained and qualified personnel.

 Risk of explosion: Do not use this product in the presence of inflammable anaesthetics or gases.

 Always examine the product for damage before commencing treatment. Damaged accessories or a damaged unit must not be used and must be replaced. Use original EMS spare parts and accessories only.

 Have this product exclusively repaired by an approved EMS repair center.

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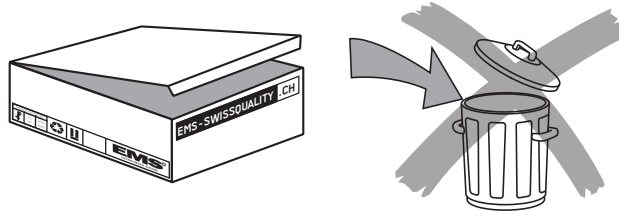
## STORING THE PRODUCT

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 Keep the original packaging until the product is to be disposed of permanently. You can use it for shipping or storing your unit at any time.

Should you wish to put your product out of use for an extended period of time:

- Proceed as described in the chapter "Cleaning, disinfecting and sterilizing"
- Pack the product and all accessories in the original packaging

Please refer to the storage and transport conditions in the "Technical data".

 The powder should not be stored near acids or heat sources.

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## PRODUCT DISPOSAL

The product must not be discarded in domestic household waste.

Should you wish to definitively dispose of the product, please comply with the regulations which apply in your country.

## WARRANTY

Damages due to non-adherence to the operating instructions or wear out of parts are excluded from warranty.

 The warranty of your product will be cancelled if you try to open it.

## ACCESSORIES

Accessories are available from EMS or any authorized dealers. Please contact your customer service directly.

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## EMS-SERVICE

Should your product need additional servicing or repairs, please send it to your dealer or our approved EMS repair center.

In the case of non-authorized repairs or damaged due to non-adherence to the operating instructions, EMS accepts no liability whatsoever. This will also void the warranty.

It is best to ship your product in the original packaging. It protects your product against damage during shipment.

Before dispatching your product including all accessories, please clean, disinfect and sterilize as described in the operating instructions.

The connector may be removed or replaced only by an approved EMS repair center.

You can modify the type of adapter on the device within the available range at an approved EMS repair center only.

When sending your product directly to the approved EMS repair center, please include the name of your dealer. This simplifies processing for us.

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## TECHNICAL DATA

Description	
Manufacturer	EMS Electro Medical Systems SA CH-1260 Nyon, Switzerland
Model	AIR-FLOW® handy 2+
Classification 93 / 42 EEC	Class IIa
Mode	Continuous operation
Service Pressure to the turbine connection : Water	0.7 to 2.2 bar (700-2200 hPa) with a service flow of 50-80 ml/min.
Service Pressure to the turbine connection : Air	Static pressure 3.5 to 4.5 bar (3500-4500 hPa)
Weight	Approx. 0.160 kg
Operating conditions	+10° C to +40° C 30% to 75% relative humidity 500 hPa to 1060 hPa air pressure
Storage and transport conditions	-10°C to +40°C 10% to 95% relative humidity 500 hPa to 1060 hPa air pressure

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# TROUBLESHOOTING

Type of problem	Solutions
Water comes into the powder chamber or leaks from the cap	<p>Check the dental unit connection</p> <p>Check the state of the O-rings of the turbine connection</p> <p>Clean the cap and the powder chamber</p> <p>Clean the handpiece</p> <p>Connect the unit to the dental unit (be careful, the powder chamber must be empty)</p> <p>Check the quality of the spray</p> <p>Fill the powder chamber</p> <p>Send the unit to an approved EMS repair center</p>
No powder/air jet coming from the unit	<p>Cut immediately the air input releasing the foot pedal</p> <p>Wait for 1-2 minutes for the system to decompress</p> <p>Disconnect the unit from the dental unit</p> <p>Press the foot control pedal of the dental unit</p> <p>If no air comes out for the turbine connector, it is your dental unit that causes the problem</p> <p>If air comes out, it is the device that causes the problem</p> <p>Unscrew the cap with the top of the wash-hand basin, but of the powder can be ejected. A sealed apparatus, even disconnected, can indeed remain under pressure</p> <p>Empty the powder chamber and screw the cap back on</p> <p>Connect the unit to the dental unit (be careful, the powder chamber must be empty)</p> <p>Connect the handpiece to the unit</p> <p>Press the foot control pedal of the dental unit</p> <p>If air comes through the handpiece connector, the handpiece is blocked. Clean the handpiece</p> <p>If no air comes out of the handpiece connector, the unit is blocked</p> <p>Send the unit to an approved EMS repair center</p>
Air and/or powder is leaking from the threads of the cap	<p>Check the seal and the cleanliness of the thread on the powder chamber and on the cap</p> <p>Replace the seal if necessary</p>
The unit efficiency diminishes	<p>A powder refill is perhaps necessary</p> <p>Clean the handpiece</p> <p>Send the unit to an approved EMS repair center</p>

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e-mail: [info@ems-ch.de](mailto:info@ems-ch.de)

SPAIN

EMS Electro Medical Systems España SL

Bernardino Obregón 14 bis

E-28012 Madrid

Tlf. +34 91 528 99 89

Fax +34 91 539 34 89

e-mail: [ems-es@arrakis.es](mailto:ems-es@arrakis.es)

FRANCE

EMS France Sarl

23, Av. Louis Bréguet

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F-78140 Vélizy Villacoublay

Tél. +33 1 34 58 03 80

Fax +33 1 34 58 03 90

e-mail: [info@ems-france.fr](mailto:info@ems-france.fr)

ITALY

EMS Italia S.p.a

Via Faravelli 5

I-20149 Milano

Tel. +39 02 3453 8111

Fax +39 02 3453 2778

e-mail: [dental@ems-italia.it](mailto:dental@ems-italia.it)

USA

EMS Corporation

11886 Greenville Avenue #120

Dallas, TX 75243, USA

Tel. +1 972 690 83 82

Fax +1 972 690 89 81

e-mail: [emsusa@emsdent.com](mailto:emsusa@emsdent.com)

**WWW.EMS - SWISSQUALITY.COM**