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**Artikel**

Harvard MTA Universal HandMix, 1g Pulver, 3ml Flüssigkeit, Dosierlöffel

**Instructions for Use EN**
**Endodontic repair cement based on mineraloxides**

Harvard MTA Universal HandMix is a biocompatible, endodontic repair cement. The powder consists of fine hydrophilic particles of various mineral oxides. After contact with the liquid it forms a gel that hardens to an impermeable barrier.

**Indications**

- Repair of root perforations
- Root-end filling (retrograde)
- Pulp capping (direct and indirect)
- Apexification / Root-end filling (orthograde)
- Repair of internal resorption
- Pulpotomy

**Contraindications / side-effects**

None known

**1. Dispensing and Mixing**

The **powder/liquid ratio is 2.6/1.0**. This can be obtained by mixing 1 level (orange) scoop of powder and 2 drops of liquid.

If a thinner or firmer consistency is desired, the mixing ration can be modified slightly:

Mixing ratio (powder/liquid by weight)	2:1	2,6:1	3:1
Working time from start of mixing (at 23 °C/74 °F)	3:00 min	2:00 min	1:00 min

For mixing of Harvard MTA Universal HandMix use a mixing pad that is impervious to water or a glass block.

For accurate dispensing of Harvard MTA Universal HandMix powder shake the bottle to loosen the powder. Overfill the spoon with the powder, level the powder with the mixing spatula and carry it onto the mixing pad.

For dispensing of Harvard MTA Universal HandMix liquid turn the bottle vertically with the tip about 5 cm above the mixing pad. Steady your hand and squeeze the bottle gently to dispense one drop at a time. If any bubbles are present, lightly tap the bottle with the fingers holding it. **Discount undersized drops and drops with bubbles. Discount also oversized drops**, usually resulting from holding the bottle too close to the mixing pad or squeezing the bottle too hard and/or too long.

Use a small spatula to rapidly mix all the cement powder in portions into the liquid. The mixed cement should be thixotropic and have a homogeneous consistency. **Total mixing time is 30 seconds.**

If desired, a more rigid consistency is achieved by adding some more powder to the mixture, a more creamy consistency is attained by adding some liquid.

**After use, tightly close both liquid and powder bottles to prevent exposure to moisture.**

**2. Application**
**2.1. Repair of root perforations**

Place rubber dam and clean the root canal system using intracanal instruments and irrigate with NaOCl. Dry the root canal with paper points and isolate the perforation. Fill the root canal apical of the perforation completely with a suitable root canal filling material.

Mix Harvard MTA Universal HandMix as described under point 1.

Apply Harvard MTA Universal HandMix with suitable instruments into the perforation site and condense it. Check the position of Harvard MTA Universal HandMix in the root canal by an X-ray. If an adequate barrier has not been created, rinse the MTA cement out of the canal and repeat the procedure.

Remove excess moisture with a cotton pellet or a paper point.

Place a moist cotton pellet in the access to the root canal and apply a temporary filling material. Alternatively seal the root canal with a suitable root canal filling material and seal the cavity with a tight filling. Both options can be done at the earliest **5 minutes** after placement of the Harvard MTA Universal HandMix.

Harvard MTA Universal HandMix repair material remains as a permanent part of the root canal filling.

**2.2. Root end filling (retrograde)**

Create a surgical access to the root-end and resect the root. Prepare an apical cavity to a depth of 3-5 mm. Isolate the area and dry the root-end cavity with paper points. Achieve hemostasis with suitable methods. Mix Harvard MTA Universal HandMix as described under point 1.

Apply Harvard MTA Universal HandMix with suitable instruments and condense it using a small plugger.

Remove excess cement and clean the surface of the root with a moist piece of gauze.

Confirm placement of Harvard MTA Universal HandMix repair material with an X-ray. Harvard MTA Universal HandMix remains as a permanent part of the root canal filling.

**2.3. Pulp Capping**

Place rubber dam and prepare the cavity. Rinse the cavity and exposed pulp areas with a suitable disinfectant.

Mix Harvard MTA Universal HandMix as described under point 1.

Apply a small amount of Harvard MTA Universal HandMix with a suitable instrument over the exposed pulp and remove excess moisture with a cotton pellet. At the earliest **5 minutes** after application of Harvard MTA Universal HandMix place a small amount of a flowable light cure liner (e.g. Harvard IonoLine) and light cure.

Etch the remaining cavity walls according to the total-etch-technique with Harvard Etch and apply a suitable bonding agent (e.g. Harvard Bond TE Bond) according to the corresponding instructions.

Place a light cure composite (e.g. Harvard PremiumFill) according to the instructions and light cure. Check pulp vitality and status regularly.

**2.4. Apexification / Root-end filling (orthograde)**

Place rubber dam and clean the root canal system using intracanal instruments and irrigate with NaOCl. Dry the root canal with paper points.

For disinfection place calcium hydroxide paste in the root canal and seal the access opening with a temporary filling material.

Remove the calcium hydroxide paste after one week from the root canal system.

Mix Harvard MTA Universal HandMix as described under point 1.

With a suitable instrument apply a small amount of Harvard MTA Universal HandMix into the apical region and condense it. Create a 3 – 5 mm barrier of MTA cement.

Check the position of Harvard MTA Universal HandMix by an X-ray. If an adequate barrier has not been created, rinse the MTA cement out of the canal and repeat the procedure.

At the earliest **5 minutes** after application of the Harvard MTA Universal HandMix place a cotton pellet in the access to the root canal and apply a temporary filling material.

Alternatively seal the root canal with a suitable root canal filling material and seal the cavity with a tight filling. Harvard MTA Universal HandMix remains a permanent part of the root canal filling.

**Additional remarks**

- In the first hour after application handle the placed MTA cement carefully.
- Intraoral application of Harvard MTA Universal HandMix must be done immediately after mixing to prevent dehydration during setting.
- MTA cement can cause discoloration.

**Storage**

Store Harvard MTA Universal HandMix at a dry place at 10 - 25 °C (50 °F - 77 °F). **Do not store below 10°C (50 °F)!** Do not use after expiry date.

**Warranty**

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

**Limitation of Liability**

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

**For dental use only!**
**Store product out of reach of children!**

Keep the instructions for use for the duration of the application.

**Order-No.**

7081507

**Article**

Harvard MTA Universal HandMix, 1g powder, 3ml liquid, dosage spoon

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