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MUPPY® designs MUPPY® standard is available in two different sizes:

- with red ring for deciduous dentition (size I)
- with blue ring for mixed dentition (size II)

There are also different colours and hardness grades (elastic, rigid) available.

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MUPPY® Standard
elastisch (nur in transparent)
elastic (only transparent)

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Instructions for use Before using MUPPY® for the first time, clean it thoroughly under warm running water. Check MUPPY® regularly for places that have been bitten through! Replace MUPPY® when necessary.

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Cleaning instructions Clean thoroughly with a toothbrush under warm running water. Alternatively, you can also use the Dr. Hinz Orofan® dental cleaning sticks. MUPPY® is neither dishwasher- nor microwave-safe. Clean the hygienic MUPPY® Box only with a damp cloth.

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*For the early weaning of sucking habits in malpositions
that are not well developed*

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MUPPY® for prophylaxis *Even if there is still no well-developed malposition, the damaging sucking habit can be ended with the help of MUPPY®. MUPPY® is offered to children as substitute, which they voluntarily accept and wear because it is not forced on them.*

The easy MUPPY® standard also helps with faulty (infantile) swallowing, tongue habits like lip biting and sucking or serves to switch from mouth to nasal breathing.

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The wearing calendar can be downloaded from dr-hinz-dental.de/muppy

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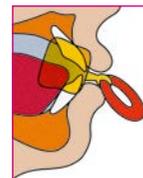
MUPPY® rules *Studies have shown that more than 90 % of children give up damaging sucking with the help of MUPPY®. Your support and a few rules help your child:*

- 1 *MUPPY® is a substitute for thumb and dummy. The substitution must be voluntary.*
- 2 *MUPPY® must always be worn at night.*
- 3 *To get used to it easier children should wear MUPPY® also a few hours during the day (when playing or watching television).*
- 4 *As a rule, weaning is successful within a week. However, MUPPY® should also be worn for 6–8 weeks after successful weaning of the dummy ("recidivism").*
- 5 *MUPPY® can be stored in the hygiene box.*
- 6 *Do not forget to regularly brush the teeth and MUPPY®!*

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How is MUPPY® worn? *MUPPY® is worn similarly to a common weaning dummy, lying loosely behind the lips and in front of the teeth, in the so-called oral vestibule and is held automatically by the lip muscles.*

If your child is having speech therapy treatment, you should take note of the following:

- *Follow your speech therapist's individual wearing and exercise instructions exactly.*
- *Support your child during this stage, as he/she has to feel the importance of mouth closure in order to take the exercises seriously.*
- *Make your child aware of the "rules of play" and supervise the necessary exercises.*