



Cavex Oral Pre Rinse - GB

Product Description:

Cavex Oral Pre Rinse contains 1,5% hydrogen peroxide. The oxygenating action of the hydrogen peroxide works as a bleaching agent. It reduces the total amount of microorganism and removes discolorations on the teeth.

Indications:

Cavex Oral Pre Rinse can be used as a pre procedural rinse before starting the actual dental procedure (i.e. whitening)

Ingredients:

Aqua, erythritol, glycerol, hydrogen peroxide, poloxamer, PVM/MA copolymer, mint aroma, sodium saccharin

Directions:

Cavex Oral Pre-Rinse is a ready-to-use rinse aqueous solution. Patients 18 years of age and older rinse 10 – 15 ml for at least 30 seconds, then thoroughly spit out. Always follow directions from dentist or dental professional.

Safety warning:

For professional use only.

Avoid contact with eyes.

Rinse eyes immediately if product comes into contact with them.

Contains hydrogen peroxide

Do not swallow

Content:

500 ml aqueous solution.

Storage:

Unopened bottles: 1 year when refrigerated.

After opening: 1 month at room temperature.



Manufacturer:

Cavex Holland BV

Fustweg 5

2031CJ Haarlem

The Netherlands

Tel +31 23 530 77 00

dental@cavex.nl

www.cavex.nl

Versie: 30-04-2020