POLA

FOR ALIGNERS


ADVANCED TOOTH
WHITENING SYSTEM

**WELCOME
TO YOUR NEW
POLA SMILE**

PROFESSIONALLY
USED AND
RECOMMENDED

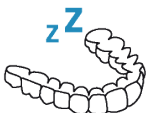SAFE FOR
OVERNIGHT WEAR

EASY TO USE

FLUORIDE
RELEASING

INDEX

ENG - ENGLISH	4-5
BUL - BULGARIAN	6-7
CRO - CROATIAN	8-9
DAN - DANISH	10-11
DEU - DEUTSCH	12-13
NLD - DUTCH	14-15
SPA - ESPANOL	16-17
EST - ESTONIAN	18-19
FIN - FINNISH	20-21
FRA - FRANÇAIS	22-23
ELL - GREEK	24-25
ITA - ITALIANO	26-27
LAV - LATVIAN	28-29
LIT - LITHUANIAN	30-31
NOR - NORWAGEIAN	32-33
POL - POLSKI	34-35
POR - PORTUGUESE	36-37
ROM - ROMANIAN	38-39
SLO - SLOVAKIAN	40-41
SWE - SWEDISH	42-43
TUR - TURK	44-45

INSTRUCTIONS FOR USE FOR PATIENTS AND DENTAL PROFESSIONALS

POLA NIGHT 10% CARBAMIDE PEROXIDE ADVANCED TOOTH WHITENING SYSTEM

Pola Night is a high viscosity, neutral pH take home tooth whitening gel. It is a pleasantly flavoured, crystal clear sustained release gel with a unique blend of soothers, conditioners and a water content high enough to further reduce sensitivity.

INDICATIONS FOR USE

- Whitening of discolored vital teeth.
- Whitening of discolored non-vital teeth.

CONTAINS HYDROGEN PEROXIDE

10% carbamide peroxide = 3.6% hydrogen peroxide


TREATMENT GUIDELINE

Pola Night 10%: 1 x 2 hours / day to overnight

Note: Patients with sensitivity will require a day or two break between treatments. Shortening treatment time is also suggested.

PLACEMENT PROCEDURE:

1. Place a small drop of gel into every compartment of the aligner for the teeth undergoing treatment.
2. Seat the aligner, with the gel around the teeth.
3. Wipe away excess gel in mouth with a tissue or dry soft brush.
4. After treatment, remove aligner. Rinse aligner and mouth with lukewarm water.
5. Brush teeth.

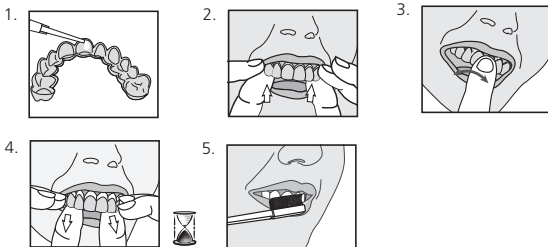FOR EFFECTIVE RESULTS

- Due to discomfort or sensitivity that can occur as a result of your orthodontic treatment, avoid using this product for the first two weeks after receiving your orthodontic aligners and for the first two days after each new aligner set, as whitening may increase tooth sensitivity.
- Do not eat, drink or smoke during treatment.
- Do not smoke immediately after treatment - wait for at least two hours.
- Foods and drinks containing strong colours should be avoided for at least 48 hours after the treatment period or consumed in moderation.
- Use gel at room temperature.

STORAGE AND SHELF LIFE

- **Pola Night** - Store in a cool and out of direct sunlight at temperatures between [2°-25°C / 35°-77°F]
- Do not use after expiration date.

PRECAUTIONS

- To be used under the supervision of a dentist / orthodontist.
- Not to be used on a person under 18 years of age, pregnant or lactating women.
- To be only sold to dental practitioners.
- For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use.
- Keep out of reach of children and pets.
- Patients with a history of chemical allergies are advised to carry out allergy testing by a specialist before using these products.
- Discontinue use if any unusual sensitivity or reactions occur and consult your dentist.
- Do not inject Pola Night into the body.
- CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A DENTIST.
- SDS available at www.sdi.com.au or contact your regional representative.

FIRST AID

Eye (contact): Hold eyelids apart and flush with running water for at least 15 minutes. Seek medical advice if symptoms persist.

Skin (contact): Wash thoroughly with water and seek medical advice if symptoms persist.

Ingestion (large amount): If swallowed, do NOT induce vomiting. Rinse mouth. Drink plenty of milk or water and seek medical advice.

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SDI Limited
SDI Germany GmbH