EN	DIRECTIONS FOR USE .....	3
FR	PROTOCOLE D'UTILISATION.....	6
IT	ISTRUZIONI PER L'USO .....	9
DE	GEBRAUCHSANWEISUNG .....	12
ES	INSTRUCCIONES DE USO .....	15
NO	BRUKSANVISNING .....	18
CS	NÁVOD K POUŽITÍ .....	21

ProRinse®

EN

## **DIRECTIONS FOR USE**

### **PERIODONTAL / ENDODONTIC IRRIGATING PROBES**

#### **1) INDICATIONS FOR USE**

These instruments / products have to be used only in a clinical or hospital environment, by qualified users.

Application field: ProRinse® is indicated to irrigate periodontal pockets and root canals.

#### **2) CONTRAINDICATIONS**

None known.

#### **3) WARNINGS**

Sterile unless tamper proof seal is broken and individual cartridge is opened or damaged. Examine individual cartridge pack for integrity of seal prior to use. Do not use if tamper proof seal is broken or cartridge is opened or damaged.

**Do not bend probe more than 25° during use.**

**SINGLE USE ONLY.** If used more than once there is a cross contamination risk to the user and patient. **DO NOT STERILIZE BEFORE USE.** The product is sterile, if resterilized, the probe may separate from the hub during use.

#### **4) PRECAUTIONS**


To prevent needle sticks, keep probe covered with protector until ready to use. Retain sheath to remove needle after use and dispose in approved sharps container.

#### **5) ADVERSE REACTIONS**

None known.


## 6) STEP-BY-STEP INSTRUCTIONS

- 1) Grasp needle cartridge and twist. Tamper evident seal will break.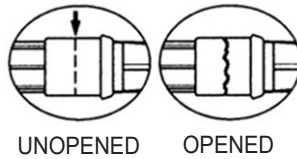


**! IMPORTANT!**  
**DO NOT BEND OR SNAP OPEN**

- 2) After twisting, pull cap straight off sheath cartridge. Keep probe covered with protector until ready for use.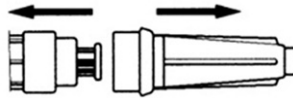

- 3) To attach needle: Grasp by needle sheath and attach to luer tip. Use a 3ml syringe. **Turn until completely secure.** Make sure tip is securely fastened before use.

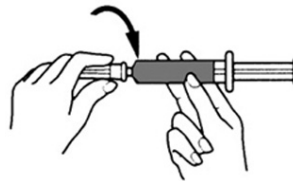- 4) Remove sheath and retain to remove needle after use.  
5) After use dispose in approved sharps container.

## 7) DISINFECTION, CLEANING AND STERILIZATION

Not applicable.

## 8) SHELF LIFE

Shelf life of product, 5 years from date of manufacture.

Symbols	EN
	For single use only
	Caution: see directions for use
	Opened packages are not replaced
	Do not use if seal broken
	Sterilized by radiation
	Sterilized using ethylene oxide
	Manufacturer

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