Palavit®55 VS

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**GB** Instructions for use

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**SE** Bruksanvisning

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CE 0197

## GB Instructions for use Palavit®55 VS

**Product description**  
Palavit 55 VS is a selfcuring resin for provisionals and other universal applications in fixed and removable prosthetics. It consists of a powder component in a range of colours A2, A3, A3.5, B3, D3, 1 (incisal) and liquid, USA only: A1 superlight, B1 superlight, B10, B, D, C, 20

**Composition**  
Powder contains:  
Methylmethacrylate-Methacrylate-Copolymer  
Liquid contains: Methylmethacrylate

- Indications**
- Repair of facings
  - Repair of methylmethacrylate-crown and bridge veneers
  - Luting of laboratory fabricated facings (on metal frames)
  - Fabrication of long-term temporaries using denture teeth
  - Attachment of denture teeth to RPD frameworks
  - Custom modification of denture teeth
  - Fabrication of replacement facings

**Contraindications**  
Do not use:  
– for intraoral application if a dry field is unattainable or the material cannot be processed properly  
– Palavit 55 VS unpolymerised directly on dentine or enamel and do not let polymerise when in contact with tooth structure (polymerization heat)  
– If any known allergies exist to one or more components of Palavit 55 VS the product should not be used.

**Note**  
The product contains (meth)acrylate compounds that may cause sensitisation by skin contact. In case of irritation or known allergies to resins meth / acrylate the product should not be used.

**Side effects**  
System induced side effects are unknown. Occasional allergic reactions were reported.

**Interaction with other materials**  
Contact with cationic mouth washes or plaque removers and hexachlorophene may cause discolorations.

**Mixing**  
Palavit 55 VS liquid is placed in the mixing cup with the eye dropper. The powder is then added and homogeneous mixed until the mixing ratio of 1,2 ml liquid to 2 grams of powder is reached. The two components are mixed for approx. 20 seconds with a glass or modelling spatula until the dough gels to a creamy consistency and hardy drops of the mixing instrument. The quantity of material mixed depends on the particular job to be done.

### Mixing and polymerisation chart

23°C (73°F)	Mixing Stage 20 sec	Pouring Stage 40 sec	Plastication Stage 3 min	Polymerisation Stage 4 min
	1 min →	4 min →	ca 8 min →	

37°C (99°F)	Mixing Stage 20 sec	Pouring Stage 40 sec	Plastication Stage 2 min	Polymerisation Stage 1 min
	1 min →	3 min →	4 min →	

Changing the mixing ratio will influence the polymerisation time. Very thin layers (luting of facings) may take up to 20 minutes for complete curing.

### Repair of facings

A matching shade is to be selected and the metal surface is prepared (mechanical retentions), then cleaned and dried carefully. Palavit 55 VS liquid is applied to the metal surface and the powder/liquid mix is prepared. The dough is applied in layers and the tooth contour is modeled by avoiding air bubbles. After polymerisation, the facing is finished with a medium diamond and pre-polished with a

silicone wheel by applying very little pressure. High gloss is achieved with a rag wheel and regular polishing pastes.

**Repair of methylmethacrylate-crown and bridge veneers**  
Prior to repair, the surface is cleaned and the matching shade selected. The surface is roughened to provide mechanical retentions. Palavit 55 VS liquid is applied to the surface and the powder/liquid mix is prepared. The dough is applied in layers and the tooth contour is modeled by avoiding air bubbles. After polymerisation is completed, finishing and polishing is done in the usual manner (see above).

**Luting of laboratory fabricated facings (on metal frames)**  
After placing a rubber dam, the facing is tried and adjusted as necessary. In order to enhance the bond, the framework and the facing surface are roughened with a medium diamond. Palavit 55 VS liquid is then applied to the roughened, cleaned and dried surfaces. Powder/liquid mix is prepared and applied to the bonding surface of the facing. The facing is then firmly pressed onto the metal surface until the polymerisation of Palavit 55 VS is completed. Excess is removed with a medium diamond and the facing is polished in the usual manner (see above).

**Fabrication of long-term temporaries using denture teeth**  
Pre-op models are first prepared by the technician; model rings are exposed, matching acrylic denture teeth are selected and their moulds prepared. The moulds are filled until they fit on the prepared abutments. The teeth are set up on the model with wax and a matrix is then formed with Formasil® Xact laboratory silicone putty. The cured matrix is removed, the teeth and the model are splashed off and the stone cast is isolated with Aislar®. The lingual surfaces of the hollowed denture teeth are roughened by sandblasting or with a medium grit stone or diamond. The teeth are repositioned in the matrix, some glue may be used to hold them in place. Palabond® or Palavit 55 VS liquid is applied onto the roughened surfaces and the matrix is put back on the model and held down with rubber bands. Palavit 55 VS is mixed and then poured into the tooth loaded matrix to form the lingual side of the temporary. The polymerisation takes place in the Palamat® practi at 55°C (131°F) and 2 bar (30 psi) pressure for 15 minutes. The matrix and the temporary are removed from the model. After polymerisation remove excess with middle fine diamonds and polish without pressure with silicone polishers and buff. High gloss is achieved by cotton wheel and polishing paste.

**Attachment of denture teeth to RPD frameworks**  
The denture teeth are waxed up on the RPD framework while placed on the master model. A Formasil Xact matrix is taken and then everything is splashed off. The model is isolated with Aislar and the teeth's base surfaces are roughened and the teeth repositioned in the matrix. Palabond® or Palavit 55 VS liquid is applied onto the roughened tooth surfaces and the tooth loaded matrix is repositioned on the model. Palavit 55 VS powder and liquid are mixed and poured into the matrix; polymerisation takes place in the Palamat® practi (see above). After polymerisation remove excess with middle fine diamonds and polish without pressure with silicone polishers and buff. High gloss is achieved by cotton wheel and polishing paste.

**Custom modification of denture teeth**  
The surface to be modified is roughened by sandblasting or with a medium diamond and Palabond® or Palavit 55 VS liquid is applied. Palavit 55 VS powder / liquid mix is prepared and the modified tooth contour is modeled by applying liquid to the surface. After polymerisation is completed, the modified denture tooth is finished and polished in the usual manner and then integrated in the prosthesis.

**Fabrication of replacement facings**  
The facing to be replaced is cleaned and the teeth on the master model are isolated with Aislar. Palavit 55 VS is mixed. The dough is applied in layers and the tooth contour is modeled by avoiding air bubbles. After the polymerisation in the Palamat® practi, the facing is trimmed and finished with diamonds and crosscut carbide burs while

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remaining on the model. For the final polish in the usual manner (see above), the facing is removed from the model. After cleaning in the ultrasonic cleaner, the bonding surface is roughened by sandblasting or with a medium-coarse diamond.

**Advice**  
When using Palavit 55 VS in the fluid resin matrix technique, the dough can be mixed to a rather runny consistency; however it must be polymerised in the Palamat® practi / Palamat® elite at 55°C (131°F) and 2 bar (30 psi) pressure for 15 minutes. – Powders of different base shades can be mixed to achieve individual aesthetics. – Bonding surfaces must be roughened by sandblasting with 50 micron aluminium oxide or with a medium grain diamond, do not use crosscut carbides for surface roughening.

**Storage advice**  
Do not store above 25°C (77°F). Do not use the material after expiry date. Avoid direct exposure to sun. Reseal carefully after removing product. Keep away from children. Visually inspect for damage before first use. Damaged products must not be used.

**Hazard/safety information**  
**Liquid (methyl methacrylate)**  
Highly flammable liquid and vapour. Causes skin irritation. May cause an allergic skin reaction. May cause respiratory irritation. Keep away from heat/sparks/open flames/hot surfaces. – No smoking. Use explosion-proof electrical/ventilating/lighting/equipment. Avoid breathing dust/fume/gas/mist/vapours/spray. IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower. Store locked up.  
**Powder (dibenzoyl peroxide)**  
May cause an allergic skin reaction. Avoid breathing dust/fume/gas/mist/vapours/spray. Wear protective gloves/protective clothing/eye protection/face protection. Specific treatment (see on this label). Wash contaminated clothing before reuse. If skin irritation or rash occurs: Get medical advice/attention. F.O.N SKIN: Wash with plenty of water.

Safety data sheets and more information are available at our website [www.kulzer.com](http://www.kulzer.com)

® = registered trademark of Kulzer GmbH  
\* = registered trademark of Vita-Zahnfabrik, Bad Säckingen

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© = marca comercial registrada da Kulzer GmbH