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R-SI-LINE® MONO S MATIC

**Hersteller**

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R-SI-LINE® MONO S MATIC, Order-No.: MMK1053

R-SI-LINE® MONO S MATIC FS (fast set), Order-No.: MMK1053FS

R-SI-LINE® MONO S MATIC (- FS) is an one phase impression material for functional impressions, one phase impressions and impressions for crowns, bridges, inlays, partial and complete dentures and denture repairs and impressions for prosthetic restorations.

R-SI-LINE® MONO S MATIC (- FS) is an automatically mixed, medium viscosity, addition curing precision impression material on vinyl polysiloxane base. The 5:1 cartridge can be used with commercial mixing devices.

R-SI-LINE® MONO S MATIC (- FS) is very thixotropic and firm (S). It stays in the position on the tray but under pressure it flows.

R-SI-LINE® MONO S MATIC (- FS) is super hydrophilic and thus improves the wettability and shows an excellent detail reproduction.

R-SI-LINE® MONO S MATIC (- FS) meets the requirements of ISO 4823 Type 2 (medium consistency - medium bodied).

Recommended procedures**1. Preparation of the tray**

It may be used perforated or non-perforated metal or plastic trays. In order to improve the adhesion of R-SI-LINE® MONO S MATIC (- FS) to the tray, the R-SI-LINE® SILICON-ADHESIVE (Order-No.: SIA1070) should be applied as a thin even film.

2. Insertion and preparation of the 5:1 cartridge

Insert the R-SI-LINE® MONO S MATIC (- FS) cartridge according to the instruction of the manufacturers and lock it. First remove the original cap (do not re-use!) from the cartridge. As a check press out a small amount of material without mixing cannula. Then screw on the mixing cannula R-SI-LINE® MIXING CANNULA MATIC BR (Order-No.: MIK1057).

Do not use the first extruded 1 cm of the mixed impression material.

3. Impression taking

The automatically mixed impression material is loaded directly into the preparation tray. Insert the loaded tray into the mouth within 2 min 15 s (- MONO S MATIC FS within 1 min 30 s) after start of mixing.

Maintain the position of R-SI-LINE® MONO S MATIC for at least 3 min (- FS at least 2 min) in the mouth. Times mentioned for 74 °F. Higher temperatures shorten, lower temperatures prolong these times.

4. Casting

The impression should be stored at normal room temperature (not above 78 °F) and can be poured 30 min after removal from the mouth. There are no other time limitations. There is no need for separating materials.

5. Disinfection

The impressions can be disinfected with standard disinfection solutions (HIV ...).

Special storage and/or handling conditions

The filled MIXING CANNULA functions as a seal until the next application. Do not re-use the cartridge cap. Closing the cartridge by turning on the original cap between use could cause cross contamination of basis and catalyst and thus premature setting.

Important: Check the openings of the cartridge and clean if necessary before install a new mixing cannula.

Shelf life see label. Do not use after expiry date.

Do not store above 25 °C (78 °F)! A cool storage (refrigerator) prolongs shelf life. Avoid exposure to sunlight.

The cartridge should be stored horizontally or diagonally within the device but not vertical.

Pay attention to the material safety data sheets (MSDS >> r-dental.com).

Technical data

Mixing time (MONO S MATIC, - FS):	automix
Working time incl. mixing time (MONO S MATIC):	< 2 min 15 s
Working time incl. mixing time (- FS):	< 1 min 30 s
Recommended time in mouth (MONO S MATIC):	≥ 3 min
Recommended time in mouth (- FS):	≥ 2 min
Time (Start mixing - withdrawal of mouth) (M S MATIC):	≥ 5 min 15 s
Time (Start mixing - withdrawal of mouth) (- FS):	≥ 3 min 30 s
Dimensional change after 24 hours:	< 0,1 %
Elastic recovery:	> 99,6 %
Strain in compression:	about 3,0 %
Shore-A-Hardness:	about 62

R-SI-LINE is a registered trademark of R-dental Dentalerzeugnisse GmbH.

Caution: For dental use only. US Federal Law restricts this device to sale by or on the order of a dentist or licensed professional, to use or to order the use of this device.

The described product must be used according to the instructions for use and is determined for dental use only. There is no liability for deflections resulting from inappropriate use or not intended purposes. The user is responsible for testing the suitability of the product for any purposes not explicitly stated in the instructions for use.

If there are damages to this product within the warranty period, R-dental is obliged to replace or to repair this product. There is no further claim. Outside of the liability by law there is no liability of R-dental for any loss or damages arising from this product, independently whether these damages are direct, indirect, special, incidental or consequential.

© 2012 R-dental Dentalerzeugnisse GmbH. All rights reserved.