R-SI-LINE® SILIKON-ADHÄSIV

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Hersteller:

R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg
T 040-30707073-0 • F 040-30707073-73 • I www.r-dental.com

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REF SIA1070 (R-SI-LINE® SILICONE-ADHESIVE)**Description and indication**

R-SI-LINE® SILICONE-ADHESIVE is an universal adhesive for adhering A- and C-silicone impression materials and impression trays made of metal or individualized methacrylate impression trays.

R-SI-LINE® SILICONE-ADHESIVE inhibits an uncontrolled separation of the impression material from the bottom of the tray and deformation during removal without distortion.

Note: R-SI-LINE® SILICONE-ADHESIVE can be used restricted with reduced adhesion on plastic impression trays. R-SI-LINE® SILICONE-ADHESIVE can not be used for polyether impression materials.

R-SI-LINE® SILICONE-ADHESIVE is available in a 10 ml-bottle with an application brush.

Recommended procedures**1. Preparation of the impression tray**

R-SI-LINE® SILICONE-ADHESIVE will be applied before the impression taking in a thin layer onto the inner areas of the impression tray.

After drying 3 min the impression tray can be loaded with impression material. Then the impression can be taken.

2. Cleaning of the impression tray

The R-SI-LINE® SILICONE-ADHESIVE on the tray can be removed easily using acetone.

Attention: Apply R-SI-LINE® SILICONE-ADHESIVE only in well ventilated rooms. Do not inhale!

Precautions to application, product life and/or storage

Avoid contact of R-SI-LINE® SILICONE-ADHESIVE with soft tissues, skin or eyes. If there is an accidental contact, flush immediately with copious amounts of water.

Pay attention to the Material Safety Data Sheet (MSDS), see www.r-dental.com.

Expiration date see label. Do not use after expiration date.

Do not store above 77 °F (25 °C)! Avoid exposure to sunlight. Keep out of the reach of children!

R-SI-LINE is a registered trademark of R-dental Dentalerzeugnisse GmbH.

Caution: For dental use only. US Federal Law restricts this device to sale by or on the order of a dentist or licensed professional, to use or to order the use of this device.

Warranty

R-dental Dentalerzeugnisse GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalerzeugnisse GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and R-dental Dentalerzeugnisse GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, R-dental Dentalerzeugnisse GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2020 R-dental Dentalerzeugnisse GmbH. All rights reserved.

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© 2020 R-dental Dentalerzeugnisse GmbH. Alle Rechte vorbehalten.
R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg • DE