⚠ WARNING

All operators must read, study, understand and comply with all warnings and operating instructions on both sides of this safety slip and those instructions provided with your power tool before using this brush.

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Safety glasses and full face shields must be worn by users and others in area. Persons within 50 or more feet may be within danger zone. See ANSI/ISEA Z87.1

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Observe all speed restrictions. Maximum RPM or Maximum Safe Free Speed (MSFS). See ANSI 3.1.8

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Tool guards must be kept in place at all times.

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Appropriate protective gloves must be worn.

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Appropriate protective clothing must be worn.

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Encima

Sur

Wear respiratory protection against dust and fumes (see ANSI Z88.2).

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Comply with the Safety Standard ANSI B165.1 "Safety Requirements - Power Brushes". Inspect brushes before mounting for damage, rust or other type of deterioration. Jog to partial speed for one minute before increasing to operating speed. During this time no one is to be in front of or in line with the brush. Never exceed Maximum RPM on the Brush.

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