# SPEEDO-CLEAN®

---

Bedienungsanleitung  
Instruction Manual  
Mode d'emploi

---

# Speedo-Clean

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# Speedo-Clean

## 1. Instructions for the user

- 1.1. Speedo Clean complies with modern technique and the basic requirements of the EC directives.
- 1.2. The user is responsible for the compliance of all safety instructions and for the specified use. Everyone using the unit should know how to use Speedo-Clean and be able to operate the unit without any danger.

## 2. Safety instructions

- 2.1. Intended use: Speedo-Clean has been developed and manufactured for the aesthetic surface cleaning of dentures in order to remove visible defects, i.e. discolouration. Note: The daily cleaning and disinfection of the dentures carried out by the patient using specific cleaning agents to maintain its functionality cannot be replaced by the unit.
- 2.2. Possible danger:  
Since the Speedo-Clean unit is operated with a magnetic field, all objects and units that could be influenced negatively by the magnetic effect should be kept out of a radius of 50 cm distance (pace maker, watches, etc.).  
Keep the lid of the cleaning tank closed during operation. If Speedo-Clean Liquid is used, the relevant safety instructions must be observed.

## 3. Technical data

Output:	35 watt
Speed:	1.700 rpm
Voltage:	230 V
Frequency:	50/60 Hz

### 3.1. Function of Speedo-Clean:

The unit removes discoloration and helps to provide a new shine to metal surfaces without damaging the model. Stainless steel pins in the cleaning tank rotate due to a magnetic field. By adding the cleaning solution Speedo-Clean Liquid, the concerned model becomes effectively cleaned.

## 4. Installation and operation

- 4.1. Installation:  
Take Speedo-Clean out of the transport box and find a suitable non-slip surface to install the unit (also see 2.2. possible danger). Please inspect the unit on possible damages. Connect the unit to the mains (230 V).
- 4.2. Operating devices:
  1. Main switch (on the back of the unit)
  2. Power supply (on the back of the unit)
  3. Timer incl. START/STOP switch

- 
- 4.3. Operation:
- Actuate the main switch on the back of the unit – the display flashes
  - Set the time on the clock "MIN." (through 99 minutes)
  - Actuate the START button – the device keeps running for the adjusted time, from 1 minute, the remaining time is indicated in seconds
  - Intermediate stop – press STOP button, continue with START button
  - End of operation time is indicated acoustically – has to be confirmed by actuating the START/STOP button
- 4.4. Cleaning of dentures, metal framework, etc.:
- Fill approx. 50 % of the enclosed steel pins into a cup.
  - Put the object into a cleaning box (for dentures upper and lower jaw only) and fill it with water so that the object is evenly covered.
- Before cleaning miscellaneous works such as telescopes etc. these should be padded with cotton wool or similar material so that the friction is not affected.**
- Add one or two caps of Speedo-Clean Liquid to the water, quantity as necessary.
  - Put the closed container onto the unit and set the timer (approx. 30 – 45 min.).
  - Rinse off the cleaned object with water afterwards
- 4.5 Cleaning of the container:
- The attached magnet is for fixing the cleaning needles when discharging the container. Hold magnet underneath the container and pour out liquid.
- 5. Maintenance and care**
- The unit does not require any maintenance.
  - Just wipe off the unit with a moist cloth.
  - Please check the plastic cup regarding density regularly.
  - Liquid must not run over/into the device.
  - In case acidic cleaning solutions are used, the needles have to be thoroughly cleaned under water.
  - Needles and cup can be sterilized in an autoclave at 134°C. For this purpose, put needles on a tray with paper lining.
  - The needles have to be replaced when the length is only still approx. 4 mm.
- 6. Guarantee**
- The producer ensures that the product is free of defects and that it shows all guaranteed features and/or all characteristics agreed upon. The warranty starts with the day of delivery of the product and is valid for twelve months. Excluded from the warranty are faults, careless use of the product, connection to unsuitable power supply, operation with the wrong voltage, acts of God, such as fire, lightning, humidity, etc. In case of justified complaints the producer has the right to repair, to deliver spare parts or to decrease the sales price. Cancellation of the sales contract can only be claimed by the customer after two repairs were carried out without success.

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HAGER & WERKEN GmbH & Co. KG