tempofit® duomix

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tempofit® duomix

Instructions for use

Material for temporary crowns and bridges, self curing, normal setting, bis-acrylic-composite 5:1

Indications for use:

- Construction of crown and bridge temporaries
- Short- and long-term temporaries

Contraindication:

- In very rare cases, **tempofit® duomix** may cause sensitization. Should such reaction occur, discontinue the use of this material.

Technical Data:

- **Mixed weight:** 25 g (Dispenser)
- **Mixing ratio:** 5 : 1
- **Product colours:** A2 / A3 (as indicated on package)
- **Mixing time:** approx. 30 sec.
- **Application time:** approx. 30 sec.
- **Time in mouth:** approx. 2 min. – 2 min. 30 sec. (hard-elastic phase, from beginning of mixing)
- **Setting time:** approx. 5 – 6 min. (until setting is completed)
- **Compressive strength:** 180 Mpa
- **Flexural strength:** 75 Mpa
- **Transverse strength:** > 32 Mpa
- **Barcol hardness:** > 28
- **Water uptake:** 10 µg/mm³
- **Maximal temperature reached during setting:** < 40 °C / 104 °F
- **Application:** At 23 °C ± 2 °C / 73 °F ± 4 °F
- **Storage:**

Ordering information:

tempofit® duomix Starterkit 02763
 2 dispenser of 25 g, A2
 1 dispenser of 25 g, A3
 1 mixing pad
 1 application syringe
 1 spatula

Single-Bag
 1 dispenser of 25 g
 1 application syringe
 1 mixing pad, 1 spatula
A2 02106
A3 02270

Standard packing
 4 dispensers, each 25 g
A2 02795
A3 02762

Eco-packing
 8 dispensers, each 25 g
A2 02800
A3 02872

Application-syringe
 10 pcs. **02802**

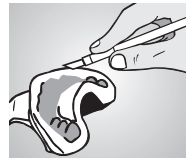
Fig. 1

1. Preparation of impression

Before any preparation is done (stump or bridge preparation, tooth extraction), make a situation impression using alginate or silicone. Interdental ridges should be excised to increase the amount of material available for finishing later (Fig. 1). If there is a gap between molars, a connecting bar between prepared teeth is created by cutting a groove between the abutment teeth in the impression. Alginate impressions should be kept at 100% rel. humidity until filling with **tempofit® duomix**.

Fig. 2

2. Use of the dispensing gun

Open the dispenser by snapping off the cap (Fig. 2). Extrude a small amount of material (bleeding) to insure proper flow from both orifices (Fig. 3). Dispenser is now ready for individual mixing (Fig. 4).

Use enclosed extra cap to close orifices again. There are guide grooves in the closure cap to avoid contamination by improper positioning of the closure cap.

Fig. 3

3. Mixing and application

Mixing of base and catalyst occurs by pressing onto piston in the dispenser. For each temporary unit extrude a length of 1 cm onto mixing pad (Fig. 5). Mix both components with spatula within 20 – 30 sec. homogeneously. Apply **tempofit® duomix** to deepest area of the impression first and fill up to gingival level (Fig. 6). Before cleaning of the application syringe **tempofit® duomix** must have hardened.

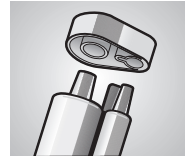
Fig. 4

4. Modelling of the temporary

Seat filled impression onto prepared teeth and remove any excess material with plastic instrument. After 2 min. – 2 min. 30 sec. **tempofit® duomix** will attain a hard-elastic consistency and can be removed together with the impression. Be sure to closely follow the polymerisation process, as removal is only possible while material is still elastic (Fig. 7).

Fig. 5

5. Hardening and finishing

Remove temporary from impression and remove excess material and proximal undercuts. After 5 – 6 min. the acryl temporary is completely cured and hard enough to be trimmed with rotating instruments. Contact with oxygen creates an inhibition layer on the temporary which should be removed by ethyl or isopropyl alcohol.

Fig. 6

6. Fastening the temporary

tempofit® duomix temporaries can be fastened using commercially available temporary dental cements eg **tempolink®** (eugenol-free). Please note that any residues of eugenol containing cements must be completely eliminated after removal of the temporary from the mouth, as they may interfere with the setting process if composite-type cements are used for subsequent attachment.


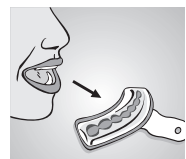
Fig. 7

7. Repairing and making minor corrections

Repairs or reinforcement can be performed by means of **smartrepair®**. For aesthetic adaptation of the gingival areas, use **smartrepair® rosa**. Polymerisation is effected by light curing, using a hand lamp. Thoroughly clean and dry temporary. Roughen area to be treated and be sure to remove any residual dust from grinding. Apply **smartbond®** evenly using **smartbrush** and distribute with circular movements. Use air syringe to spread **smartbond®** layer evenly; let dry for 1 min. Apply **smartrepair®** and expose all treated areas to UV light for at least 40 sec. Finish curing process by exposing all areas to light for additional 30 sec. Repaired areas can then be trimmed and polished.

Precautions

- Do not manually remove the resin components.
- Avoid contact of unmixed components with skin or mucosa. Wear suitable eye protection and gloves!
- Only for intended use by qualified personnel.
- Please follow the instructions of the safety data sheet!

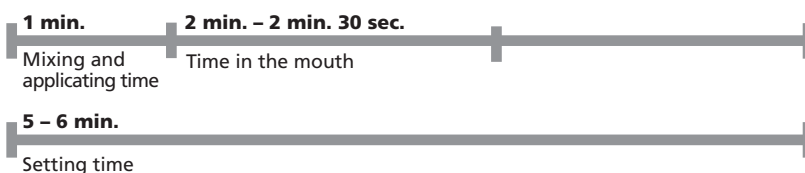DETAX

GmbH & Co. KG
 Carl-Zeiss-Str. 4 · 76275 Ettlingen/Germany
 Telefon: 0 72 43/510-0 · Fax: 0 72 43/510-100
 www.detax.de · post@detax.de

Caution: Federal U.S. law restricts this device to sale by or on the order of a dentist (or trained specialist personell).

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