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### RECOMMENDED INDICATIONS

- Tissue conditioning.
- Temporary relining.
- Functional impression taking.

### CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to methacrylate monomer, methacrylic polymer, ethanol or aliphatic acid ester-based plasticizer.

### DIRECTIONS FOR USE

#### 1. PREPARATION

Check the patient's oral conditions and the denture in use. Adjust misfitting areas with a carbide bur and polish dry until sufficient space so GC TISSUE CONDITIONER can be applied to the denture to maintain its thickness in order to maintain the elasticity. Apply COCOA BUTTER or petroleum jelly to the areas where the denture should not be relined.

#### 2. POWDER/LIQUID DISPENSING

The powder is provided in a 2.4 g/2 mL (1 unit of powder to 2 units of liquid). Dispense the required amounts of powder and liquid into the rubber cup.

Note: More liquid will result in a shorter gel phase and a more viscous mixture. More liquid will result in a longer gel phase and a less viscous mixture.

Do not mix any form aggregates in the bottle under certain storage conditions. This will not affect the function.

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3. MIXING

Mix powder and liquid for 30-60 seconds until the mixture reaches a creamy consistency.

d) Apply to denture.

4. THE COATING AGENT

The coating agent can be used as a bonding agent to improve the bond strength and to avoid possible delamination. Apply the coating agent with the brush only on the areas where a stronger bond is needed. In this case, GC TISSUE CONDITIONER can be peeled off by hand.

5. APPLY TO DENTURE

Note: Avoid sticking of the material to the instruments or instruments with lotions.

a) Prevent from flowing in the throat.

3) Seat the denture in the mouth. Ask the patient to gently occlude in the centric position and move the teeth and mandibles to muscle trim. After keeping the denture in the mouth for 3 minutes in this position, remove and inspect for excess and deficiencies.

4) Remove any excess using a sharp instrument.

5. RINSE AWAY SALIVA

Wash away saliva from the relined denture under running water. Dry with an air syringe.

6. APPLY THE COATING AGENT

Apply the coating agent to the remaining surfaces of the denture. Dry with a sharp instrument thoroughly with an air syringe or leave it undisturbed for 4-5 minutes.

Note: To improve the marginal seal, coating agent can also be applied to the GC TISSUE CONDITIONER / denture interface.

7. RESEAT THE DENTURE IN THE MOUTH.

Note: An impression of the denture in water before seating in the mouth will reduce unwanted odor.

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Care should be taken to use denture cleaning agents for a prolonged period of time.

### 6. REMOVAL

It is recommended to remove the material or replace it with a new one after days or weeks depending on each clinical case.

GC TISSUE CONDITIONER can be peeled off by hand. Use a sharp instrument or brush to remove the area where the coating agent was used as a bonding agent.

### 7. STORAGE

Recommended for optimal performance, store in a cool, dark place (4-25°C) (39.2-77.0°F) away from direct sunlight and high temperature.

### 8. COLORS

Live pink, White

### PACKAGES

1. 1-Packaging

Kit contains: Powder 90g (1), Liquid 90g (10mL) (1).

Coating agent 12g (15mL) (1), Powder measure (1), Plastic liquid syringe No.1 (1), Brush No.7 (1), Plastic spatula (1).

Rubber cup (Large) (1)

b) GC TISSUE CONDITIONER powder 90g (in each shade)

c) GC TISSUE CONDITIONER liquid 90g (10mL)

c) GC TISSUE CONDITIONER coating agent 12g (15mL)

### CAUTION

1. The liquid is flammable. Avoid sources of ignition. Do not keep or use near heat or high temperature.

2. Use in a well-ventilated place. Replace cap immediately.

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4. Avoid contact with food, mixture or coating agent on skin.

In case of contact with skin, wipe with a cotton pellet or sponge soaked in alcohol and flush with water.

5. Coating agent may turn yellow with time, but this will not affect the function.

6. Do not use on sensitive skin.

7. In rare cases the product may cause sensitivity in some people.

If any such reactions are experienced, discontinue the use of the product and refer to a physician.

8. Personal Protective Equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present EU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available:

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### CLEANING AND DISINFECTION

MULTI-SITE DECONTAMINATION SYSTEM: to avoid cross-contamination between patients this product requires a site-specific level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged if necessary. Sterilization: The sterilization cycle to prevent drying and accumulation of organic material. Disinfection: The disinfection cycle to healthcare-grade infection control product according to regional / national guidelines.

Undesired effects: Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instrument for use, report them directly through the relevant reporting system or to the proper authority of your country accessible through the following link:

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As well as through vigilance system vigilance@gc.dental

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PRINTED IN JAPAN

EU: GC EUROPE N.V.

Responsible for Health and Safety of Goods, Interclean 33,

B-3001 Leuven, Belgium

TEL: +32 16 74 10 00

Prior to use, carefully read the instructions for use

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Prior to use, carefully read the instructions for use.

EN

**GC TISSUE CONDITIONER**

TISSUE CONDITIONING AND RELINING MATERIAL

For use by or under the supervision of a dental professional in the recommended indications.

- RECOMMENDED INDICATIONS**
1. Tissue conditioning.
  2. Temporary relining.
  3. Functional impression taking.

**CONTRAINDICATIONS**  
Avoid use of this product in patients with known allergies to methacrylate monomer, methacrylic polymer, ethanol or aliphatic acid ester-based plasticizer.**DIRECTIONS FOR USE**

1. PREPARATION  
Check the patient's oral conditions and the denture in use. Adjust misfitting areas with a carbide bur and dry. Create sufficient space so GC TISSUE CONDITIONER can be applied at a minimum thickness of 1 mm in order to maintain elasticity.

Apply COCOA BUTTER or petroleum jelly to the areas where the denture should not be relined.

2. POWDER / LIQUID RATIO  
The standard powder / liquid ratio is 2.4 g / 2 mL (1 unit of powder to 2 units of liquid). Dispense the required amounts of powder and liquid into the rubber cup. Note:

a) Using less liquid will result in a shorter gel phase and a more viscous mixture.

More liquid will result in a longer gel phase and a less viscous mixture.

b) Powder may form aggregates in the bottle under certain storage conditions.

This will not affect the function.

c) Close the bottles of powder, liquid and coating agent tightly immediately after use.

3. MIXING  
Mix powder and liquid for 30–60 seconds until the mixture reaches a creamy consistency.

4. APPLICATION  
a) The coating agent can be used as a bonding agent to improve the bond strength and to avoid possible delamination. Apply the coating agent with the brush only on the areas where a stronger bond is needed. In this case, GC TISSUE CONDITIONER cannot be applied off by hand.

- b) Apply the mixture evenly to the fitting surfaces of the prepared denture. Note:

a) To avoid staining of the material, lubricate finger tips or instruments with water. Do not use lotion or petroleum jelly.

b) Prevent excess from flowing into the throat.

c) Seal the denture in the mouth. Ask the patient to gently occlude in the centric position and move the lips and mandibles to muscle firm. After keeping the denture about 5 minutes in this position, remove and inspect for excess and deficiencies.

- d) Remove any excess using a sharp instrument.

5. FINISHING  
a) Wash away saliva from the denture under running water.  
b) Dry with an air syringe.

- c) Apply the coating agent to the relined surfaces of the denture. Dry the coating agent thoroughly with an air syringe or leave it undisturbed for 4–5 minutes.

Note:  
To improve the marginal seal, coating agent can also be applied to the GC TISSUE CONDITIONER / denture interface.

3. Reline the denture in the mouth.

Note:  
a) Immersion of the denture in water before seating in the mouth will reduce unwanted odors.

b) Instruct the patient to clean the relined areas with fingers under running water. The use of a brush may damage the relined areas.

Care should be taken not to use denture cleaning agents for a prolonged period of time.

6. REMOVAL  
It is recommended to remove the material or replace it with a new one after several days or weeks depending on each clinical case.

GC TISSUE CONDITIONER can be peeled off by hand. Use a sharp instrument or bur to remove the area where the coating agent was used as a bonding agent.

**STORAGE**  
Recommended for optimal performance, store in a cool, dark place (4–25°C) (39.2–77.0°F) away from direct sunlight and high temperature.**COLORS**  
Live pink, White**PACKAGES**  
1. 1.1 Package

Kit contains: Powder 90 g (1), Liquid 90 g (101 mL) (1), Coating agent 12 g (15 mL) (1), Powder measure (1), Plastic liquid syringe No.1 (1), Brush No.7 (1), Plastic spatula (1), Rubber cup (Large) (1)

## 2. Retail Packages

- a. GC TISSUE CONDITIONER powder 90 g (in each shade)  
b. GC TISSUE CONDITIONER liquid 90 g (101 mL)  
c. GC TISSUE CONDITIONER coating agent 12 g (15 mL)

**CAUTION**

1. Nicht entzündlich. Avoid sources of ignition. Do not keep or use near fire or high temperatures.

2. Use in a well ventilated place. Replace cap immediately.

3. Avoid contact with eyes. In case of contact, immediately flush with water and seek medical attention.

4. Avoid direct contact of liquid, mixture or coating agent on skin. In case of contact, wipe with a cotton pellet or sponge soaked in alcohol and flush with water.

5. Coating agent may turn yellow with time, but this will not affect its function.

6. Do not mix with other products.

7. In rare cases the product may cause sensitivity in some people. If any adverse reactions are experienced, discontinue the use of the product and refer to a physician.

8. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gc-europe.com>

They can also be obtained from your supplier.

**CLEANING AND DISINFECTION**

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.

DO NOT IMMERSIVE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

Undesired effects- Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, please report them to the manufacturer's instruction for use, please report directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

[https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en)

In this way you will contribute to improve the safety of this product.

Last revised: 08/2019

'GC.'

MANUFACTURED by  
GC DENTAL PRODUCTS CORP.

2-285 Tomioka-cho, Kasugai, Aichi 486-0844, Japan

DISTRIBUTED by  
GC CORPORATION

1-1 Nakajima-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU : GC EUROPE N.V.

Reservoirstraat 1200, 3000 Leuven, Belgium TEL: +32 16 74 00

GC ASIA DENTAL PTE. LTD.

5 Tamplines Central 1, #06-01 Tamplines Plaza, Singapore 529541

GC AUSTRALASIA DENTAL PTY. LTD.

1753 Botany Rd, Bankstown NSW 2200, Australia

TEL: +61 2 9301 8200

PRINTED IN JAPAN

Prior to use, carefully read the instructions for use.

EN

**GC TISSUE CONDITIONER**

TISSUE CONDITIONING AND UNFETTERING

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