UNIFAST™ Trad

Prior to use, carefully read the instructions for use.

EN

GENERAL PURPOSE ACRYLIC RESIN

GC UNIFAST Trad is a self curing, fast setting acrylic resin for use in a variety of dental applications. For use in either pressure or paint-on techniques.

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

- Repair of dentures.
- Construction of temporary crowns.
- Impression trays for individual teeth.
- Orthodontic plates, splints, check bite and other general purposes.

CONTRAINDICATIONS

In susceptible individuals, sensitization to the product cannot be excluded. Use of the product should be discontinued if allergic reactions are observed.

DIRECTIONS FOR USE

A. PRESSURE TECHNIQUE

The recommended powder / liquid ratio is 1g of powder (1st graduation of powder measure) to 0.5mL (1st graduation of liquid measure) of liquid.

- Dispense the required amount of liquid into the rubber cup as follows:
 - Remove the protective cap and screw on the dispensing nozzle (fig. 1).
 - Pour into the measure provided (fig. 2) until the bottom of the meniscus is level with the required graduation (fig. 3).
 - Replace the protective cap to prevent evaporation.
- Add the powder and mix thoroughly for 10-15 seconds.
- The mixture reaches a dough-state 20-30 seconds after mixing. All manipulation should be finished before 2 minutes after mixing when setting starts.

B. PAINT-ON TECHNIQUE

Use standard techniques.

STORAGE

Store in a cool place away from direct sunlight. 4-25°C/39.2-77°F (3 year shelf life guarantee).

NOTES

- Do not use UNIFAST Trad powder or liquid with components of other brands of acrylic resin.
- Remove residual resin with a dry gauze after each application.
- Mop up liquid spills with a dry cloth.

PACKAGES

- Bottles of powder, 100g, 250g, 1kg.
- Bottles of liquid, 100g (104mL), 250g

SHADES

- Crown colours : Ivory.
- Gingival colours : Pink # 3, Live pink # 8, Pink X.

WARNING

- The liquid is HIGHLY FLAMMABLE. Avoid sources of ignition.
- In case of contact with eyes, flush immediately with water and seek medical attention.

Last revised: 08/2012

GC
MANUFACTURED BY
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803, U.S.A.

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UNIFAST™ Trad

GENERAL PURPOSE ACRYLIC RESIN

For use only by a dental professional in the recommended indications.
UNIFAST Trad is a self curing, fast setting acrylic resin for use in a variety of dental applications. For use in either pressure or paint-on techniques.

RECOMMENDED INDICATIONS

For repair of dentures, construction of temporary crowns, impression trays for individual teeth and orthodontic plates, splints, cheek bite and other general purposes.

CONTRAINDICATIONS

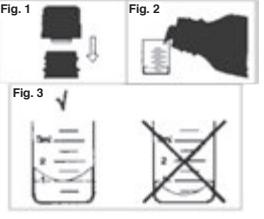
Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.

DIRECTIONS FOR USE

A. PRESSURE TECHNIQUE
The recommended powder/liquid ratio is 1g of powder (1st graduation of powder measure) to 0.5mL (1st graduation of liquid measure) of liquid.

- Dispense the required amount of liquid into the rubber cup as follows:
- Pour into the measure provided (fig. 2) until the bottom of the meniscus is level with the required graduation (fig. 3).
- Replace the protective cap to prevent evaporation.

B. PAINT-ON TECHNIQUE
Use standard techniques.

STORAGE
Recommended for optimal performance, store in a cool place away from direct sunlight (2-37°C/36-98°F).

NOTES
1. Do not use UNIFAST Trad powder or liquid with components of other brands of acrylic resin.
2. Remove residual resin with a dry gauze after each application.
3. Mop up liquid spills with a dry cloth.

PACKAGES

- Bottles of powder: 100g; 250g; 1kg
- Bottles of liquid: 100g (104mL); 250g (260mL)

SHADES

- Crown color: Ivory
- Gingival colors: Pink (#3); Live Pink (#8); Pink X

CAUTION

- The liquid is HIGHLY FLAMMABLE. Avoid sources of ignition.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Personal Protective Equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

CLEANING AND DISINFECTING RECOMMENDATION
MULTI-USE DELIVERY SYSTEMS: To avoid cross-contamination between patients the bottles and measuring devices require mid-level disinfection. Immediately after use inspect the bottles, measuring devices and label for deterioration. Discard if damaged. DO NOT IMMERSER. Thoroughly clean bottles and measuring devices to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the Safety Data Sheets available at: <http://www.gceurope.com> or for the Americas: <http://www.gcamerica.com>

They can also be obtained from your supplier.

Undesired effects- Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental. In this way you will contribute to improve the safety of this product.

Last Revised: 10/2019

MANUFACTURED BY
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0969
www.gcamerica.com

DISCLAIMER

GC CORPORATION

76-1 Hekimura-cho, Itabashi-ku, Tokyo, Japan

Researchaphare Headquarters: 1540, Itabashi-avenue 33

TEL: +81-3-3989-1915

Fax: +81-3-3989-1916

RESP. TEC. Mayara De Santos Ribeiro - CRO/SP: 105.982

PRINTED IN THE U.S.A.

Rx Only

RESPONSIBLE MANUFACTURER IN CANADA
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0969
www.gcamerica.com

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