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ZM-Bond®

CE 0482

HerstellerR-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg
Telefon 040 - 30707073-0 • Fax 040 - 30707073-73 • www.r-dental.com**ZM-Bond®**

REF ZMF4200

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REF ZMF4200

Description and indication

ZM-Bond® is a single-component primer, which promotes a chemical adhesion between luting composites and restorative materials based on metal, zirconium dioxide and ceramic.

Characteristics and composition

ZM-Bond® has chemical groups, which on the one hand include functional groups on methacrylate basis for luting composites and on the other hand functional groups for an adhesion to indirect restorative materials and frameworks. The luting composites on methacrylate basis (particularly FANTESTIC®-product line) can be self-curing with light-cure option that is dual-cure. The restorative materials may be indirect restorations or frameworks.

Recommended procedures**1. Preparation of indirect restorations or frameworks****1.1 Metal**

The adhesive contact areas of the restoration will be sandblasted according to the instructions for use of the manufacturers, until a matt surface has been achieved. Then rinse the contact areas with water or clean it in an ultrasonic device and dry with oil-free air.

Attention: Do *not* use phosphoric acid on the adhesive contact areas!**1.2 Zirconium dioxide**

The adhesive contact areas of the restoration will be prepared according to the instructions for use of the manufacturers. Then rinse the contact areas with water or clean it in an ultrasonic device and dry with oil-free air.

Attention: Do *not* use phosphoric acid on the adhesive contact areas!**1.3 Ceramic**

The adhesive contact areas will be etched with hydrofluoric acid (HF) according to the instructions for use of the manufacturers. Then rinse the prepared contact areas with water and dry with oil-free air.

2. Application of ZM-Bond®

ZM-Bond® will be applied with a MICRO BRUSH (Order-No.: MPB3981) onto the pre-treated surfaces of the restorations according to 1.1-1.3. The primer should be effect for 10 s and then dispense the solvent slightly with oil-free air.

3. Adhesive treatment of indirect restorations or frameworks

The luting composites based on methacrylate (particularly FANTESTIC® product line) can be light-cure or self-curing or dual-cure. But use separate adhesive systems for the teeth (particularly FANTESTIC® FLOWSIVE product line).

Precautions to application, product life and/or storage

ZM-Bond® contains acetone. Avoid contact of ZM-Bond® with soft tissues, skin or eyes. If there is an accidental contact with eyes, flush immediately with numerous amounts of water and contact a physician. In the case of skin contact wash with numerous amounts of water. Do not use ZM-Bond® in case of allergic reactions.

Pay attention to the Material Safety Data Sheets (MSDS >> www.r-dental.com).

Shelf life see label. Do not use after expiration date.

Do not store above 77 °F! Aceton is highly volatile. Immediately close the bottle after use. Ensure a good room ventilation. Aceton is highly flammable and irritant. Keep out of the reach of children!

FANTESTIC and ZM-Bond are registered trademarks of R-dental Dentalerzeugnisse GmbH.

Caution: For dental use only.**Warranty**

R-dental Dentalerzeugnisse GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalerzeugnisse GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and R-dental Dentalerzeugnisse GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, R-dental Dentalerzeugnisse GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2019 R-dental Dentalerzeugnisse GmbH. All rights reserved.

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ManufacturerR-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg • DE