Instructions for Use


Intra-oral Adjustment/Polishing for Zirconia and Lithium Disilicate

- 1 Suitability for use**

Visually check the polishers for cracks, chips and corrosion. Ensure that the polishing medium remains secure on the bur. Do NOT use if damaged or worn out.
- 2 Sterilization**

Instruments are shipped non-sterile. Complete cleaning and sterilization instructions may be found at www.garrisondental.com/sterilization. Autoclave the instruments at 134°C with a holding time of five (5) minutes and a drying time of ten (10) minutes. It is the responsibility of the dentist or licensed practitioner to ensure effective sterilization.
- 3 Storage**

Store the polishers in the sterilization container until required for use. Storage should be in dry, clean conditions at ambient temperature.

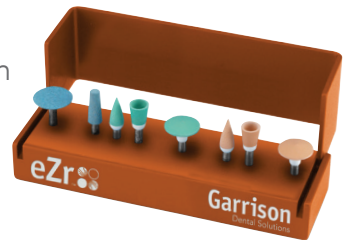4 Precautions

- A. The device is to be used by a dentist or licensed practitioner
- B. Wear gloves when handling instruments
- C. Handpieces should be maintained in good working order
- D. Ensure the polisher is securely gripped in the handpiece collet
- E. Wear eye protection to shield against particles
- F. Wear a surgical mask to avoid inhalation of particles

5 Use

- A. Maintain constant movement to avoid localized heating and potential damage
- B. The speed range for use is 7000-10,000 RPM
Do NOT exceed 20,000 RPM
- C. Avoid applying excessive pressure or leverage
- D. Best results are achieved by using polishers in order from coarse to fine grit

6 Post Use

Used burs should be considered contaminated and require appropriate handling precautions for subsequent reprocessing.

For current instructions including cleaning and sterilization, please visit www.garrisdental.com/sterilization

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