

Operating Instructions BA

1. General terms:

1.1. Product description:	(A) easy Waxer duo	415-0000	115 V	415-0115
	(B) easy Waxer	413-0000	115 V	413-0115



1.2. Delivery includes: (A) (B)

easy Waxer duo

-2 hand pieces with connection cable
-2 modelling tips PKT 1+2
-2 hand piece clamps

easy Waxer

-1 hand piece + connection cable
-1 modelling tip PKT 2
-1 hand piece clamp

1.3. Manufacturer: YETI Dentalprodukte GmbH- Industriestraße 3 - D-78234 Engen
Phone: +49-7733-9410-0 Fax: +49-7733-9410-22
www.yeti-dental.com for operating manuals
and declaration of conformity

1.4. Trade mark note: DGBM

1.5. Technical features:	Mains voltage :	230V +/- 10%		115V
	Mains frequency:	AC 50Hz		AC 50/60Hz
	Mode of adaptation / plug:	Schuko / German		US Schuko
	Power consumption:	9-12 W		9-12 W
	Fuse:	T 400 mA		T 400 mA
	Mode of protection:	safety isolation		safety isolation

1.6. Positioning and storage conditions: - make sure that the unit is horizontal and stable
- store in dry conditions at ambient temperature
- keep away from acids and bleaches

2. Uses:

YETI WAXER units are to be used **exclusively for modelling with waxes used by the dental technician. When not used according to recommendations, claims for liability cannot be accepted.**

3. Caution:

1. Ensure that only qualified dental technicians and authorised persons are allowed to use these machines. **Always read instructions before use.** Contact manufacturer if there are any queries.
2. Operate these units only in dry conditions. Do not immerse in water.
3. Caution: Do not allow the heating unit or the inserts to touch skin when hot.
4. Do not cover or pack unit when hot.
5. Any modification to the units or grips is exclusively at the user's own risk.
- 6. Any misuse of the unit will lead to loss of warranty and liability.**
7. Do not use any solvents for cleaning the unit.

Please ensure safe packing when returning a unit (e.g. for repair).

4. Operating instructions:

Dokument:	Erstellt am/von:	geändert am/von:	Revision:	freigegeben am/von:	Seitenzahl:
BA	10.10.2005/TB	25.03.2019/CJ	1	25.03.2019/TB	Seite 1 von 3

Operating Instructions BA

4.1. Before putting into operation:

YETI Waxers follow the latest technical developments and comply with standard safety regulations.

Nevertheless users are advised to pay due care and attention to the operating instructions.

- Ensure the unit is kept in a clean condition at all times.
- Observe the standard recommendations for safety at work.
- The unit should be used by skilled and authorised personnel only.
- Check the unit for technical problems - especially the cables and the housing.
- Stick the modelling tip/s required for your next work on the heating unit (push on system).
- Stick the hand piece clamp (art. no. 410-1002) onto the unit/working table and always place the hand piece there after use.



- a** – hand piece **b** – modelling tip **c** – heating unit
d – S-rings **e** – hand piece cable

4.2. Operation:

1. Put the plug into the socket (adapt only to licensed sockets - see 1.5.)
2. Turn on the switch at the back of the unit.



(A)



(B)

3. The units Easy Waxer (A+B) are ready for use after a short heating-up period. The temperature control is operated via a rotary controller. The desired temperature can be setted (A) for every handpiece separately.

Easy Waxer duo has 3 LED's for quick trouble shooting. After easy change of component the unit is within couple of minutes ready for use. If all LED's are on, the unit is fully operative.

If red LED is off (for Easy Waxer red LED in upper right corner), the transformer is damaged.

If yellow LED is off, the controller for the left or for the right handpiece is damaged.

If all 3 LED's are on but the modelling tip stays cold, the hand piece is damaged and should be replaced.

4. Dip the hot tip into the wax and place the fluid wax drop onto the wax build-up.

5. Use a suitable tool when changing the hot tip to avoid the risk of burns.

6. Turn off the unit when not required continuously and place the hand piece into the hand piece clamp to avoid the risk of fire.

(By taking away the protection foil of the adhesive tape, the clamp can be stuck both on the unit or on the work bench.)

7. Turn off the unit after use to avoid the risk of fire.

8. Wait until the unit is cold before storing.

5. Disposal: When disposing of the unit please observe the procedure operating in your country for disposal of such items.

Dokument:	Erstellt am/von:	geändert am/von:	Revision:	freigegeben am/von:	Seitenzahl:
BA	10.10.2005/TB	25.03.2019/CJ	1	25.03.2019/TB	Seite 2 von 3

Operating Instructions BA

- 6. Warranty:** In case of material or production faults, Yeti will repair the unit free of charge within 24 months from date of delivery !
Excluded from this warranty are the modelling tips and handpieces.


. Spare parts and accessories:

Our tips from left to right:

- 411-0100 PKT1 small
- 411-0200 PKT 2
- 411-0300 Hollenback
- 411-0400 Knife
- 411-0500 Corkscrew small
- 411-0600 Corkscrew thick



- 410-1002 Hand piece clamp for all Waxers included in the delivery

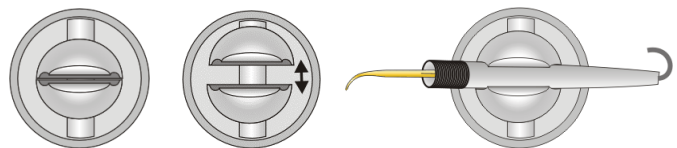

- 411-1001 Hand piece for all Easy Waxers

Rubber rings for hand piece:

- 411-4000 S-Rings for the hand piece 25 pcs./black



8. Trouble Shooting List

Unit (A) does not start and shows a black bar

Switch off and start again

Unit shows all options on the screen but the hand piece is not getting warm-

Exchange the hand piece. Parts of the hand piece are wearing parts similar to a bulb and should be occasionally replaced by the user without the repair in the manufacturing company

Dokument:	Erstellt am/von:	geändert am/von:	Revision:	freigegeben am/von:	Seitenzahl:
BA	10.10.2005/TB	25.03.2019/CJ	1	25.03.2019/TB	Seite 3 von 3