**voco**

# Ionofil Plus

In accordance with DIN EN ISO 9917-1

0482

## Instructions for use

## Product description:

**Ionofil Plus** is a radiopaque glass ionomer restorative cement with chemical adhesion to enamel and dentine.

**Ionofil Plus** is available in the shades A1, A2 and A3 and has an enamel-like translucency.

## Indications:

- For restoration of deciduous teeth (especially class I)
- Repair of V-shaped defects and enamel erosions as well as root erosions in the cervical area incl. class V fillings
- Cavities caused by carious lesions
- Fillings of class III and small fillings of class I (incl. extended fissure sealing) as well as build-up fillings for crowns, linings


## Method of use:

Creation of retentions generally is not necessary because of the good adhesion of **Ionofil Plus** to dentine. However, undercuts provide additional hold to the filling. Layer thickness of the fillings should not be less than 1 mm.

In case of carious defects prepare and clean cavity accordingly (for instance with 3 % hydrogen peroxide) and then rinse carefully with water and dry. In case of a remaining dentine thickness of more than 1 - 1.5 mm an additional lining is not necessary. A pulp protection with calcium hydroxide is recommended in cases of lesser dentine thickness.

## Shake powder well before use.

Place powder and liquid on a glass plate or a suitable mixing pad and work the powder into the liquid a little at a time to begin with, then in larger quantities. Use 1 drop of liquid for 1 level measuring scoop of powder (mixing ratio: 4.7 - 5.6 g of powder : 1 g of liquid). When dispensing the liquid, hold the dropper bottle absolutely vertically! A strong plastic spatula should be used for mixing.

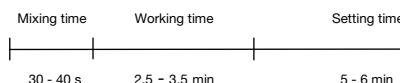The mixing time is 30 - 40 s (it should not exceed 1 min).

The working time **after** mixing is approximately 2 - 3 min (the temperature should be 15 °C - 23 °C).

After insertion of the material into the cavity, the setting time is approximately 2.5 min.

During the filling procedure make sure that the working area is kept dry.

Class V cavities should be at least 1 mm deep. Avoid feather-shaped margins. In case of doubt prepare in a conventional way.

During the setting phase, especially immediately after application, glass ionomer cements are sensitive to humidity (breath, saliva) and also to drying-out. Therefore, if possible, apply a protective varnish immediately.

After setting of **Ionofil Plus**, but no sooner than 5 - 6 min after application (depending on mixing temperature and mixing ratio) excess material may be removed with a sharp instrument and contouring can be carried out (flexible disks). Then cover again with varnish. The best results for final polishing can be obtained about 24 h after the filling has been inserted.

## Information, precautionary measures:

In the proximity of the pulp (closer than 1 - 1.5 mm) use pulp protection to prevent irritation. Close powder and liquid carefully after use in order to keep out humidity and to maintain the solid consistency. Avoid contact with the mucosa and the eyes.

**Ionofil Plus** contains fluorosilicate glass, polyacrylic acid and parabens. **Ionofil Plus** should not be used in case of known hypersensitivities (allergies) to any of these ingredients. Do not use insulating materials on dentine to maintain full adhesion (except for pulp protection).

## Storage instructions and application method:

Store at temperatures of 4 °C to 23 °C. Tightly close powder and liquid container immediately after use. Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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**VOCO GmbH**